GIMA

PROFESSIONAL MEDICAL PRODUCTS

REGGILENZUOLINI CROMATO CHROME-PLATED PAPER ROLL PORTE-ROULEAUX CHROMÉ CORREAS DE SOPORTE

MANUALE D'USO – ISTRUZIONI E MONTAGGIO
USER MANUAL – ASSEMBLY INSTRUCTIONS
MODE D'EMPLOI – INSTRUCTIONS ET MONTAGE
MANUAL DE USO – INSTRUCCIONES Y MONTAJE

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MD

INTRODUCTION

For the purposes of Regulation (EU) 2017/745, the product falls under the CLASS I MEDICAL DEVICES category. We advise you to read the following manual carefully before using the device.

DESCRIPTION

Chrome-plated paper roll holder suitable for bed legs, with a round tubular section from Ø 25 to 40 mm and square from 25x25 to 35x35 mm. (Supplied disassembled and without paper roll)

COMPONENTS

Tube to insert the roll

Fixing supports equipped with black plastic washer

INSTRUCTIONS FOR ASSEMBLY

Secure the supports to the bed legs using the plastic washer, without over-tightening. Once the correct position has been found, insert the roll and slide the tube into the special hooks. Finally, tighten the washers to secure the support to the bed legs.

TECHNICAL DATA SHEET

Size	Assembled 68x25x3.5 cm (supplied disassembled 68x10x3.5 cm)
Weight	1 kg

INSTRUCTIONS FOR USE

- Follow the instructions given in this manual carefully.
- Avoid direct contact with open wounds.
- Not suitable for outdoor use.

MAINTENANCE AND CLEANING

- Only use original spare parts.
- Periodically check all fixing points.
- To ensure its efficient, long-term use, it is advisable to avoid prolonged exposure to sun rays, contact with saline environments and storage in very damp environments.
- Wash the stainless steel surfaces with soapy water and use suitable disinfectants.
- Rinse with a damp cloth and dry carefully.
- Do not pour chlorine- or ammonia-based detergents or greasy and oily substances on the surfaces.
- Do not use metal tools for cleaning. If disinfection is required, only use suitable non-corrosive products.

	Keep away from sunlight		Medical Device compliant with Regulation (EU) 2017/745		Keep in a cool, dry place
	Product code		Consult instructions for use		Manufacturer
	Date of manufacture		Lot number		Medical Device
	Caution: read instructions (warnings) carefully		Unique device identifier		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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Gima S.p.A.