



# GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy  
Italia: tel. 199 400 401 - fax 199 400 403  
Export: tel. +39 02 953854209/221/225 - fax +39 02 95380056  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

---

## **LETTINO 3 PEZZI POLIVALENTE**

## **3-SECTION MULTIFUNCTIONAL BED**

## **DIVAN POLYVALENT 3 PIÈCES**

## **CAMILLA 3 PIEZAS POLIVALENTE**

Manuale utente - User manual - Notice d'utilisation - Manual del usuario

---

**ATTENZIONE:** *Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.*


**ATTENTION:** *The operators must carefully read and completely understand the present manual before using the product.*

**AVIS:** *Les opérateurs doivent lire et bien comprendre ce manuel avant d'utiliser le produit.*


**ATENCIÓN:** *Los operadores tienen que leer y entender completamente este manual antes de utilizar el producto.*




Gima S.p.A.  
Via Marconi, 1  
20060 Gessate (MI) Italy

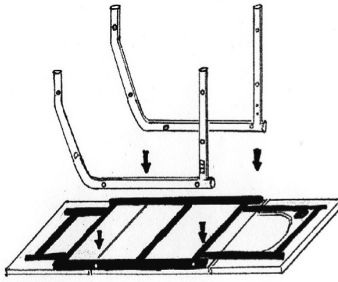


Img. 1

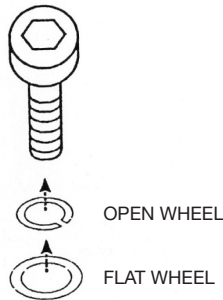


Img. 2

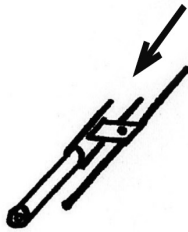

Img. 3

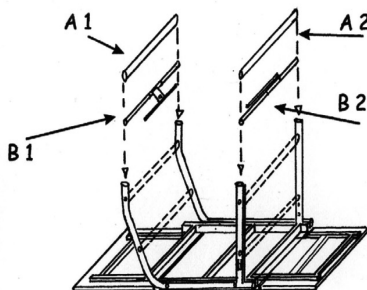


Img. 4

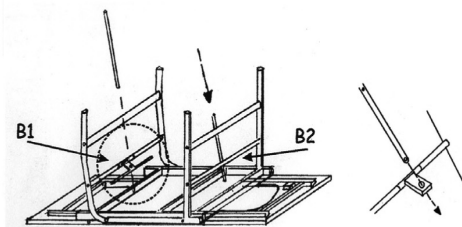

Img. 5

Img. 5.1

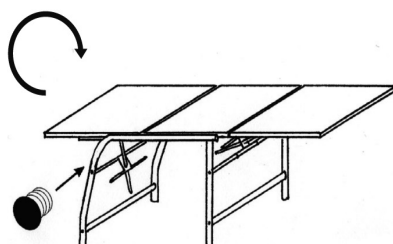

Img. 6

## ENGLISH

### Img. 1

- Pull out legs and crossbars from the box.
- Place the legs facing the threaded sleeve into the frame of the padded platform by keeping hole with bigger diameter on the external side of the bed.

### Img. 2

- Insert the screws (make sure that their wheels are correctly placed into allen screw).

Note: Screws must be firmly fixed at the end of assembly.

### Img. 3

- Insert the accessory for blocking the cushions into the chromed crossbar.

### Img. 4

- Place the crossbars as shown in the drawing:  
B1 blocking with long crossbar  
B2 blocking with short crossbar
- Insert the screws (image 2).

Note: for an optimal functioning keep chromed sleeve of b1 and b2 with bigger diameter towards the padded frame.

### Img. 5

- Insert the long chromed rod in B1 and the short rod in B2 (Images 5 and 5.1).  
Fasten the rods to the cushions with proper screws after checking that the block accessory (Image 3) faces the center of bed.

### Img. 6

Note: To place the bed in working position is strongly recommended to ask help of another person as item cod. 27505 is quite heavy.

- Fasten the screws firmly
- Insert 12 plastic head covers with a hammer

Note: the rotation of bed could cause sudden movements of the cushions, please be careful.

