



# GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy  
Italia: tel. 199 400 401 - fax 199 400 403  
Export: tel. +39 02 953854209/221/225 - fax +39 02 95380056  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

---

# **PORTAROTOLO UNIVERSALE**

# **UNIVERSAL ROLL HOLDER**

# **PORTE-ROULEAU UNIVERSEL**

# **PORTAROLLOS UNIVERSAL**

**Manuale utente - User manual - Notice d'utilisation - Manual del usuario**

---

**ATTENZIONE:** *Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.*

**ATTENTION:** *The operators must carefully read and completely understand the present manual before using the product.*

**AVIS:** *Les opérateurs doivent lire et bien comprendre ce manuel avant d'utiliser le produit.*

**ATENCIÓN:** *Los operadores tienen que leer y entender completamente este manual antes de utilizar el producto.*



Gima S.p.A.  
Via Marconi, 1  
20060 Gessate (MI) Italy

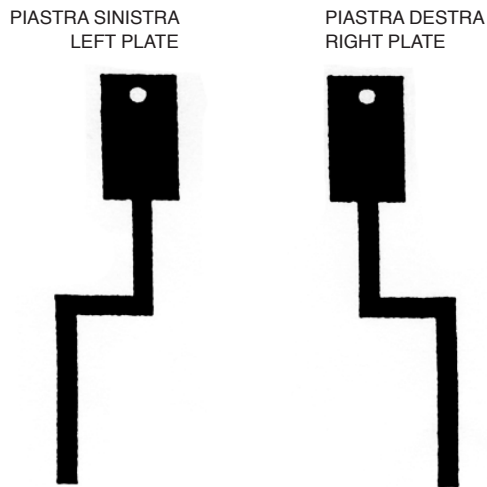


Fig. 1

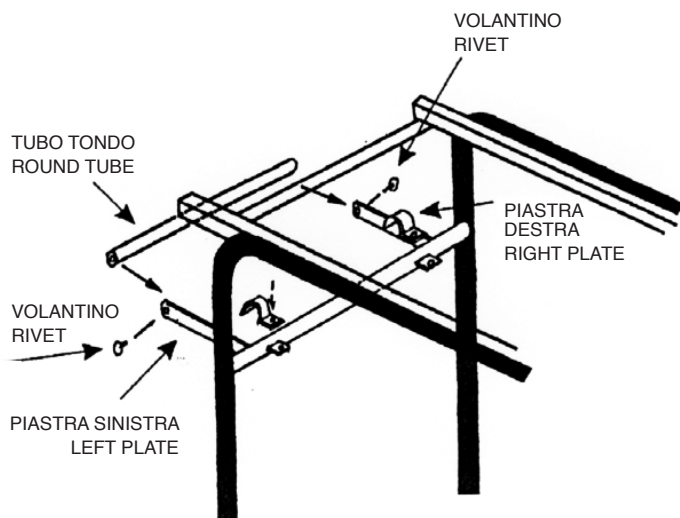


Fig. 2

**ITALIANO**

Fissare le piastre (Fig. 1) al tubo tondo come illustrato in Fig. 2 facendo attenzione a non invertire la loro posizione.

Inserire il tubo tondo fra le piastre fissandolo con i rispettivi volantini (Fig. 2).

N.B.: durante la sostituzione del rotolo è sufficiente estrarre un solo volantino allentando leggermente l'opposto. Si consiglia di controllare periodicamente la stabilità del manufatto onde evitare un suo mal funzionamento

**ENGLISH**

Fasten the plates (image 1) to the round tube as shown in figure 2. Please pay attention not to invert their position.

Insert the round tube between the plates by fastening it with the respective rivets.

Note: During roll replacement just extract one rivet by slightly loosening the opposite one. It is recommended to check regularly the steadiness of the item in order to avoid malfunction.