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1. INTRODUCTION

They are stools for professional use, completely built in stainless steel, 360° pivoting self breaking casters, rubber-tyred.

The height can be comfortably adjusted by means of a gas pump; the back-rest is 180° revolving and depth-adjustable.

The seat is balanced density padded and Peltex-lined.

There are three models: SPD, SPD/1, SPD/A and they all differ in their specific base shape

2. INSTALLATION

- 1 – To insert pivot inside wheels
- 2- To insert wheels in the basement
- 3- To install the gas column on the basement
- 4- To install the mechanism for movement to the gas column
- 5- To insert the backrest's steel in its support and to tighten the knob
- 6- To fix the upholstery with screws

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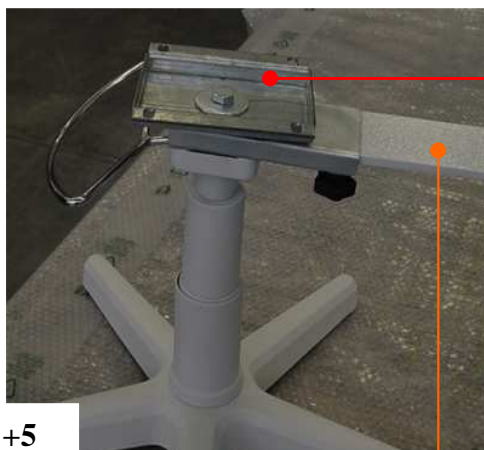
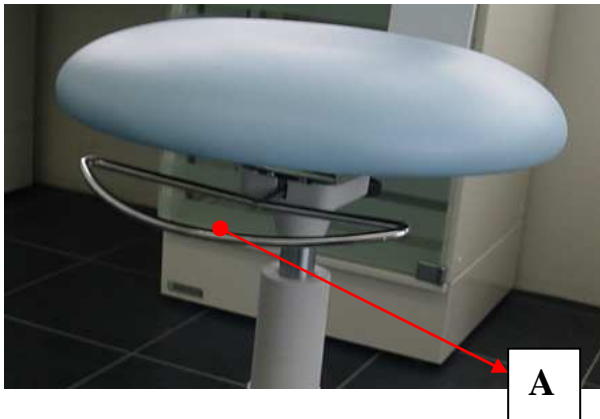
3. SEAT HANDLING

Lifting up the lever "A" the up/down movement is unblocked: to block the stool at the desired position it is enough to push down the same lever;

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
4. BACKREST HANDLING

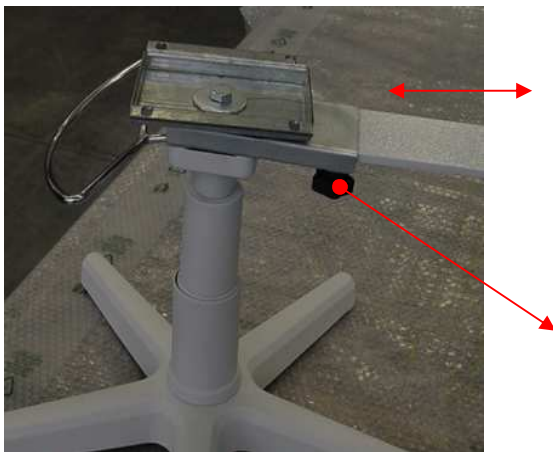It is possible to move longitudinally the backrest through knob B

5. MAINTENANCE. CHAIR CLEANING.

All materials used to construct the equipments can support cleaning and disinfection operations due in these instructions.

The great variety of drugs chemical products, which are used on medical study can damage painted surfaces and plastic material parts.

Proofs and inquiries made substantiate that surfaces cannot be outright protected by aggression of all products which you can find on the market.

The effects of these products above all depend on time and stay on surfaces.

It is important to remove that chemicals by a damp colth from equipment surfaces immediately.

To clean, please, kindly use a damp colth; do not spray the detergents directly on the equipment.

ATTENTION!

Do not use products which contain alcohol, ammonia, abrasive substances, benzene.

The concentrations listed below are the maximum ones that are consented and they cannot be surpassed:

Quaternary Ammonium: maximum 2%

Glutaric Aldehyde: maximum 2%

**EUROCLINIC WILL NOT BE HELD
RESPONSIBLE FOR DAMAGES CAUSED BY
USE OF DISINFECTANTS WHICH CONTAIN
ACTIVE SUBSTANCES DIFFERENT FROM
THOSE ABOVE MENTIONED.**

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