TITANOX s.r.l. - via Canove 2/A - Loc. Canove dé
Biazzì -26038 Torre dé Picenardi (CR) - Italy -
www.titanox.it info@titanox.it MADE IN ITALY

Instruction manual “T3000”

Structure made of epoxy painted steel tube. Three sections with independent control and Trendelenburg position. Padding made of high-density polyurethane foam, fireproof class 1 IM, covering made of washable, fireproof, plastic material (leatherette) – fireproof class 1 IM. Adjustable rubber feet. Complete with a pair of height-adjustable, padded, anatomic leg-holders, a couple of shoulder rests, handgrips, a paper roll holder and a stainless-steel basin diam 32 cm. Disassembled.

Dimensions: 181x60x80h cm. Carrying capacity: 150Kg

All models are produced eliminating sources of potential risk to the user, should unintended collisions occur, such as cutting edges or sharp corners.

These instruments are intended to be used by medical practitioners who are specially trained on how to use and care of them. Use for purposes other than what it is intended for is not allowed. The incorrect use, poor or inappropriate maintenance can rapidly lead to deterioration of the instruments.

- Check the right tightening torque of all mobile parts before and during use.
- For cleaning, do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants, and acetone: the manufacturer company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

TITANOX s.r.l. - via Canove 2/A - Loc. Canove dé
Biazzì - 26038 Torre dé Pícenardi (CR) - Italy -
www.titanox.it info@titanox.it MADE IN ITALY

TITANOX WARRANTY TERMS

Titanox 12-month warranty applies from final invoice date.

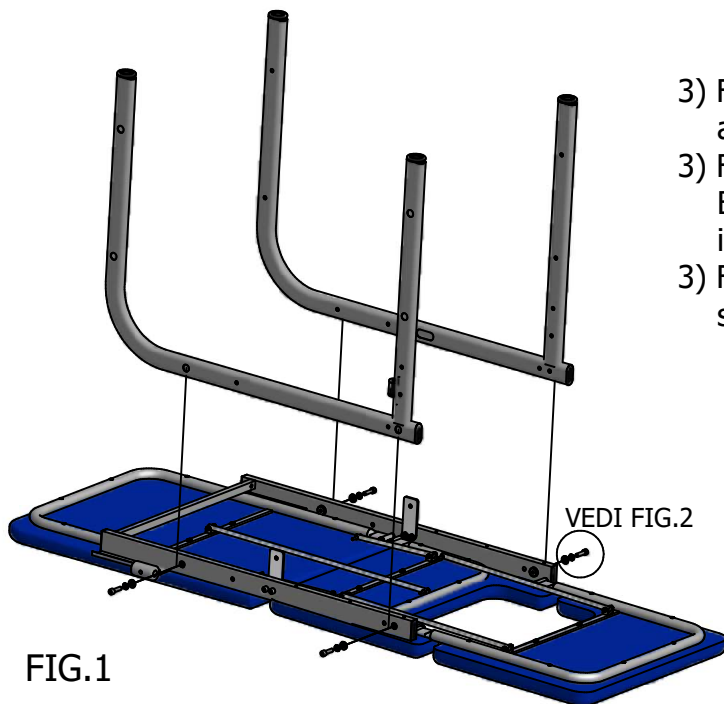
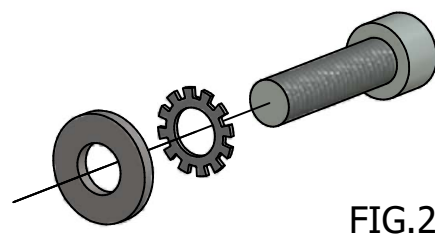
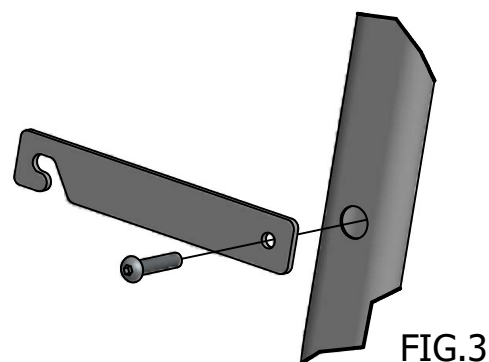
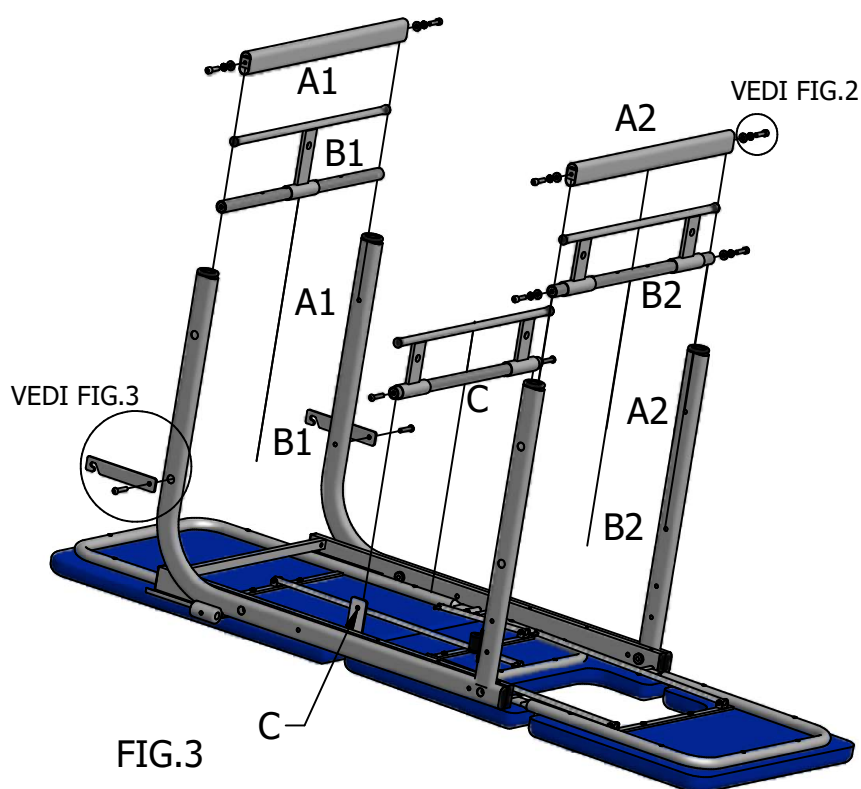
All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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