

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy Italy: tel 199 400 401 - fax 199 400 403 - gima@gimaitaly.com Export: phone + 39 02 953854209/221/225 - fax + 39 02 95380056 - export@gimaitaly.com - www.gimaitaly.com

# CODE 27557 USER'S MANUAL – DONOR CHAIR – blue





# **SYMBOLS:**

Symbol	Symbol definition
<u> </u>	Carefully read the instructions for use
类	Keep away from sunlight
<del>*</del>	Store in a dry place
CE	Product in compliance with European Directive
LOT	Lot number
<b>~</b>	Manufacturer address
REF	Product code

#### I. GENERAL FEATURES

The chair is made of oval, square and round profile section metal.

The plan for use made of high-density fire-retardant class 1 IM foam is covered with coated (vinyl) fire-retardant class 1 IM fabric. The synchronized control of the two cushions is adjustable by gas lift (600 N). The backrest angle is +60° and about -5° in the Trendelenburg position; the footboard angle is -80° and about -5°.

The armrests are equipped with an articulation that guarantees a perfect angle during use; they can be adjusted in height and, when removed, thy can be rotated of 180°. There are four adjustable feet ensuring optimal stability.

#### II. CAPACITY

130 Kg.

## **III. CLEANING**

Use common neutral detergents on the market.

### WARRANTY CONDITIONS APPLIED BY GIMA

We congratulate you for purchasing our product.

This product meets the highest quality standards in terms of material and manufacture.

The warranty is valid for 12 months from the date of supply by GIMA.

During the period of validity of the warranty, the company will take care of free repair and/or replacement of any defective parts for manufacture reasons that are well detected, with the exception of the cost of labour or any transfers, transport and packaging. The warranty doesn't include all components subject to wear.

The replacement or repair during the warranty period will not extend the duration of the warranty.

The warranty is not valid in the event of: repairs performed by unauthorized personnel or use of non-original spare parts and defects caused by negligence or misuse.

The warranty is void if the above is not respected and if the serial number (if any) has been removed, deleted or altered.

The defective products must be returned only to the retailer where the product was purchased. Products sent to GIMA will be rejected.











GIMA S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) - Italy