



GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 - fax 199 400 403
Export: tel. +39 02 953854209/221/225 - fax +39 02 95380056
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

LETTINO VISITA KING EXAMINATION COUCH KING DIVAN D'EXAMEN KING CAMILLA VISITA KING

Manuale utente - User manual - Notice d'utilisation - Manual del usuario

ATTENZIONE: *Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.*

ATTENTION: *The operators must carefully read and completely understand the present manual before using the product.*

AVIS: *Les opérateurs doivent lire et bien comprendre ce manuel avant d'utiliser le produit.*

ATENCIÓN: *Los operadores tienen que leer y entender completamente este manual antes de utilizar el producto.*



Gima S.p.A.
Via Marconi, 1
20060 Gessate (MI) Italy

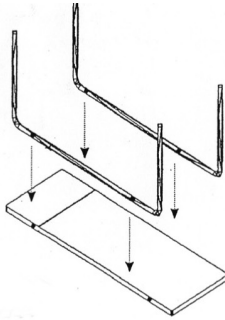


Fig. 1

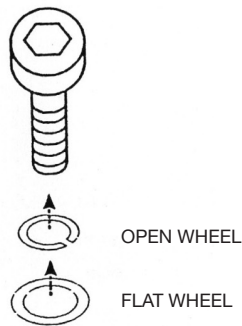


Fig. 2

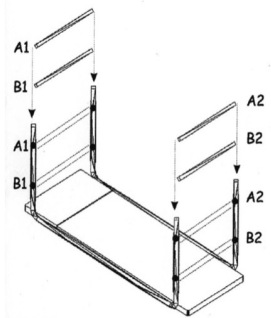


Fig. 3

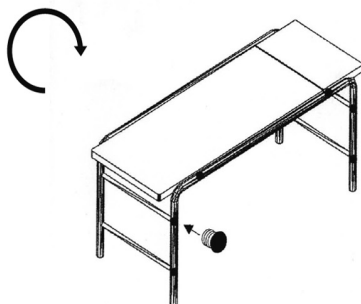


Fig. 4

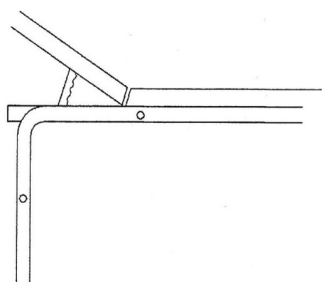


Fig. 5

ENGLISH

Img. 1

- Take out legs and crossbars from the box.
 - Set legs in front of the screwed holes into the structure of the upholstery plane (Fig. 1).

Img. 2

- Proceed with inserting of the screws - be sure that the rings of the screws are correctly inserted (Fig. 2).

Img. 3

- Insert the crossbars in the following order: B1, A1, B2, A2 as indicated.
 - Proceed with inserting of the screws (Fig. 3).

Img. 4

N.B.: Two persons are needed to put the examination couch in working position, because article is heavy. During the rotation, keep backrest in longitudinal position, in order to avoid a sudden opening.

- Tighten the screws.
 - Inset with a hammer 12 screw covers (Fig. 4).

Img. 5

Backrest is adjustable by rack. It is necessary that the rack arrives till the end of its run after positioned in the required mode, in order to be repositioned in the horizontal mode (Fig. 5).

