**LETTINI VISITA E TRATTAMENTO STANDARD E LARGE  
STANDARD AND LARGE TREATMENT TABLE  
LITS ACCOMPAGNANT ET DE TRAITEMENT STANDARDS ET LARGES  
CAMILLAS DE EXAMEN Y TRATAMIENTO ESTÁNDAR Y GRANDES  
MARQUESAS DE CONSULTA E TRATAMENTO, PADRÃO E LARGAS  
UNTERSUCHUNGS- UND BEHANDLUNGSLIEGEN STANDARD UND LARGE  
ΙΑΤΡΙΚΑ ΕΞΕΤΑΣΤΙΚΑ ΚΡΕΒΑΤΙΑ STANDARD KAI LARGE**

MANUALE D'USO – ISTRUZIONI E MONTAGGIO  
USER MANUAL – ASSEMBLY INSTRUCTIONS  
MODE D'EMPLOI - INSTRUCTIONS ET MONTAGE  
MANUAL DE USO – INSTRUCCIONES Y MONTAJE  
MARQUESAS DE CONSULTA E TRATAMENTO, PADRÃO E LARGAS  
BETRIEBSANLEITUNG – MONTAGEANWEISUNGEN  
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ – ΟΔΗΓΙΕΣ ΚΑΙ ΣΥΝΑΡΜΟΛΟΓΗΣΗ

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## INTRODUCTION

In accord with Regulation (EU) 2017/745, this product is part of the CLASS I MEDICAL DEVICES. You should carefully read this manual before using this medical device.

## DESCRIPTION

The treatment bed is intended for use in hospitals, community centers, nursing homes, physiotherapy and professional massage, visit and treatment centers.

The backrest is adjustable by manual movement. The structure is robust, removable, painted with epoxy powder; the padding is upholstery with fireproof simulated leather and colors on request.

Optional :

- Couch roll holder
- Mouth-nose plug

## COMPONENTS:

- Mattress with fireproof padding fixed to the metal frame
- N.4 legs

Assembly material:

- N. 4 10 x 70 SCREWS

## ASSEMBLY INSTRUCTIONS

The table is provided partially mounted: the mattress is already fixed to the structure. You must assembly the legs and fix the backrest mechanism.

Begin the assembly phase by setting up a stable horizontal work plan.

1. You need to assemble the legs to the structure of the padding through n.4 screws (10x70) supplied into the box

2. Fit the backrest mechanism in its place.

## TECHNICAL DATA

	<b>STANDARD</b>	<b>LARGE</b>
<b>Measurement</b>	cm. 193 x 68 x H 78	cm. 193 x 80 x H 78
<b>Weight</b>	39,5 kg	43,5 kg
<b>Max capacity</b>	180 kg	200 kg

## WARNINGS!

- Check the locking nuts are tightened.
- Observe the maximum flow rates indicated in the data sheets.
- Avoid direct contact with open wounds.
- Not for use in open environments.
- When adjusting the height of the couch and the back, make sure that no one inserts hands, legs or anything into the movement area: **CRUSHING RISK!**
- Move the bed without lifting it from the floor.
- Before moving the bed, unlock the wheels.

## MAINTENANCE AND CLEANING

- Use only original spare parts.
- Periodically check all fixing points.
- To preserve at best, it is recommended to avoid prolonged exposure to sunlight, to contact and deposit in saline and very humid environments.
- Wash the stainless steel surfaces with soapy water and use appropriate disinfectants.
- Rinse with a damp cloth and dry thoroughly.
- Do not spill on the surface cleaners containing chlorine or ammonia, fatty and oily substances.
- Do not use cleaning utensils containing iron. If necessary disinfection only use suitable non-corrosive products.

## CLEANING RULES FOR COATING:

Clean with damp soapy cloth and rinse well with clean water. Do not use solvents, bleaches, synthetic detergents, polishes or aerosol spray.

Attention: Normally synthetic leather light colours should not come in contact with no fixed cloured clothes (such as jeans and similar) not to cause stains and rings that cannot be cleaned up.

	<i>Keep away from sunlight</i>		<i>Medical Device compliant with Regulation (EU) 2017/745</i>		<i>Keep in a cool, dry place</i>
	<i>Product code</i>		<i>Consult instructions for use</i>		<i>Manufacturer</i>
	<i>Caution: read instructions (warnings) carefully</i>		<i>Lot number</i>		<i>Medical Device</i>
	<i>Unique device identifier</i>				

## GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.