



Removable Blanket Support VLA 350

Cod. 90590350



Qualifying Product **CE**

USER MANUAL

VAIMAS SRL

Via Volpato, 39
36035 Marano Vicentino VI

**AZIENDA CON SISTEMA
DI GESTIONE QUALITÀ
CERTIFICATO DA DNV GL
= ISO 9001 =**

Direttiva 93/42 CEE
Ausili Medici Classe I

Rev. 1,1

GENERAL INFORMATIONS

Aids used to lift blankets or sheets to avoid contact with the patient's body

TECHNICAL DATA AND ADJUSTMENTS

Chromed steel tube structure diameter 18 with clamps

- Height 37 cm
 - Depth 38 cm
 - East width max. 48 cm
 - Weight of structure 1.5 Kg iron, 0.70 Kg in alum.
-


ASSEMBLY

Equipment: No equipment required

Get rid of the packing the removable Blanket Support, Incase the ends of the tubes, inserting the most part

Narrow in the larger one as seen from the side to the side.

Be sure to insert the grafts until the end

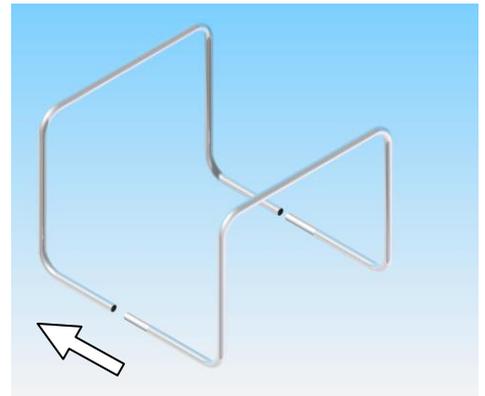

MAINTENANCE

The support does not require any special maintenance except an ordinary cleaning

USE AND CLEANING

Clean the lump with a slightly moistened cloth with normal, non-abrasive detergents.

DISPOSAL

Dispose of the device in accordance with applicable environmental and environmental regulations.

SAFETY

- MAX. 15 Kg
 - Use only for the intended uses of the manufacturer.
 - Any use other than the one provided by the manufacturer may damage the health of the patient.
 - The manufacturer does not accept any liability for damages caused by improper or unexpected use
- This manual.
- Any modification or replacement must be approved by the manufacturer.
-

WARNINGS: The information in this document is subject to change without notice.