GIMA

PROFESSIONAL MEDICAL PRODUCTS

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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- ⚠ • Do not use the product seat for a different purpose from the intended one.
- ⚠ • The bedside rails are not intended to carry the full weight of an individual.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

BEFORE USE:

- Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
- In case of damage do not use the product and contact your dealer for further instructions.
- Check, if the product is properly assembled and all screws are secure and well screwed on.
- Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment: 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign). 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed. 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

MAINTENANCE, CLEANING & DISINFECTION INFORMATION:

To guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or mild detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.
5. DO NOT leave the product wet.
6. Do not use chemical cleaners to clean the frame. This could cause damage to the surface of the product
7. If you need to disinfect the product, use a common, mild disinfectant
8. Protect the item from scratches, cuts and punctures.
9. Lubrication: the product has designed for a minimal maintenance; however, check every six months the lubrication on the moving & mechanical parts, this would ensure an optimal duration.

ASSEMBLING:

NOTE: The illustrations for the assembly are indicative to make the assembly easier. May will be differences from reality.

To assemble the Rails on the Bed:

1. Place the hooks on one side of the bed on the vertical part of the bed, e.g. on the headboard.
2. From the opposite end on the same side, e.g. on the footrest, extend the two hooks so can be hooked on the frame.

Weight Capacity: 75kgs

INTENDED USE:

The bedside rail is an accessory designed to prevent an accidental fall of a person from any bed for hospital or home care.

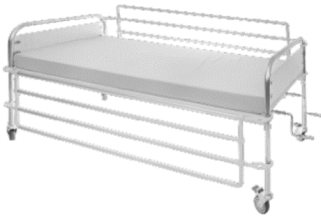
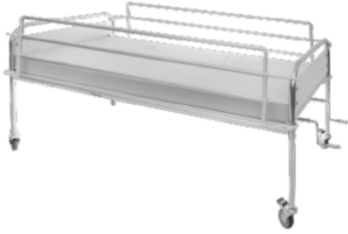
Suitable for most metal beds & with easy control system, so it does not have to be completely removed for transporting the patient to and from the bed. The bedside rails are designed to fit in most beds thank to its simple frame, that can be placed on the headboard and the footboard of the bed.

CAUTION (!):

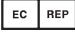
- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- LONGWAY Medical declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- LONGWAY Medical reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

GENERAL SAFETY WARNINGS:

- ⚠ • If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- ⚠ • Keep away the product from heat sources.
- ⚠ • Save this document for future reference.
- ⚠ • Be careful when children are nearby and do not allow the children to play with the product.
- ⚠ • Do not exceed the maximum payload.
- ⚠ • Special care is always required where there are moving parts that can cause limb entrapment and injury.
- ⚠ • Do not attempt to lift the product by any removable parts.

SYMBOL LEGEND

| | |
|--|--|
|  | Manufacturer |
|  | Date of manufacture |
|  | Product code |
|  | Lot number |
|  | Authorized representative in the European community |
|  | Caution: read instructions (warnings) carefully |
|  | Consult instructions for use |
|  | Medical Device compliant with Regulation (EU) 2017/745 |
|  | Medical Device |
|  | Unique device identifier |
|  | WEEE disposal |
|  | Imported by |

WARRANTY:

The product is guaranteed for a period of one (1) year from the purchasing date.

The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

 **Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.