




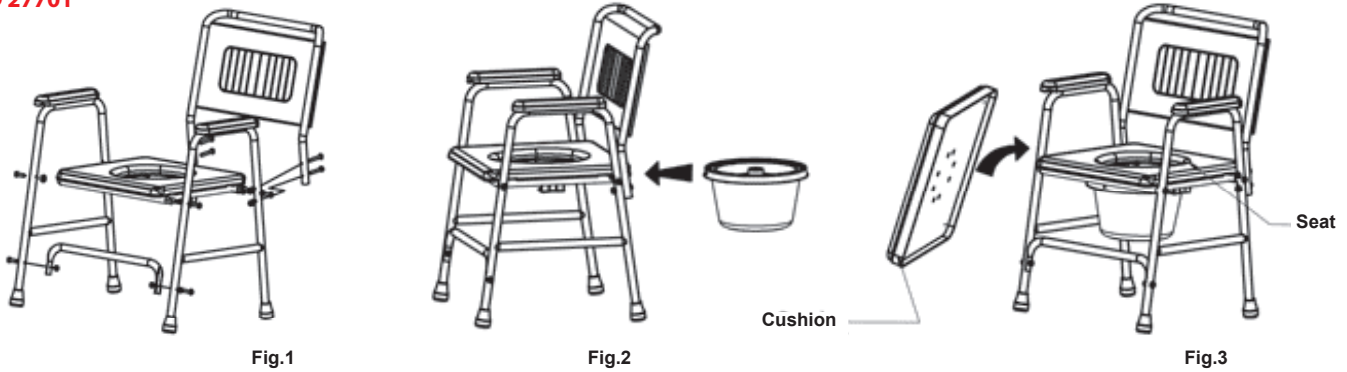
GIMA

PROFESSIONAL MEDICAL PRODUCTS

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

 You have purchased an aid designed to facilitate disabled users. Any other use is prohibited. Use only in a seated position and never stand up, for assembly refer to the explanatory images below.


COD 27701



Assembly instructions

Fig. 1. Open the package and assemble the chair as shown in the figure. Secure with screws and bolts.
 Fig. 2. Insert the toilet bowl in the direction indicated by the arrow.
 Fig. 3. Cover with the appropriate cushion after use.

COD 27702 - 27703

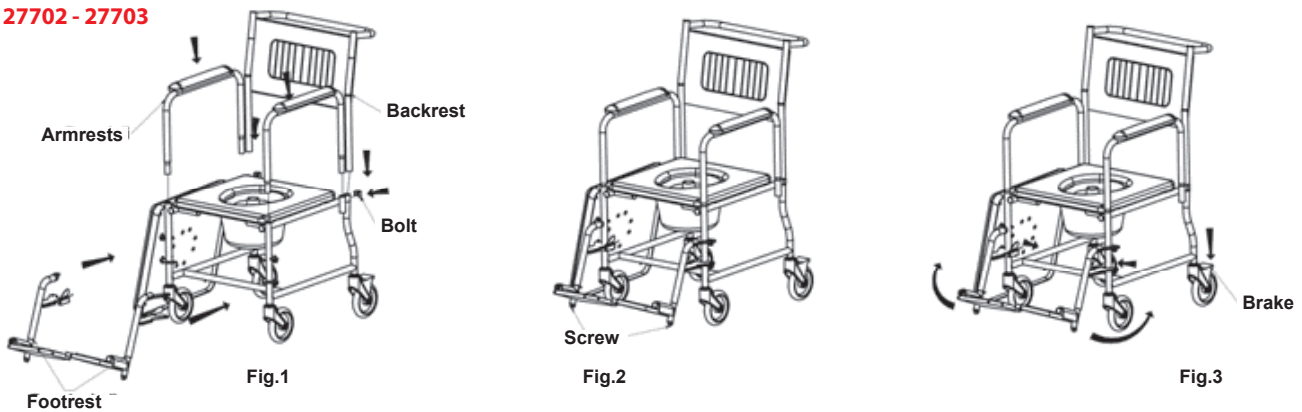

Assembly instructions

Fig. 1. Open the package and insert the armrests and footrests and the backrest on the main body of the frame. Fasten the backrest with the bolt.
 Fig. 2. Adjust the height of the footrest by the screw according to the user's need and then tighten the screw.
 Fig. 3. Before use please push down the brake of the wheels, rotate the footrest as Fig. 3 until the footrest is locked.

Safety and maintenance

1. Before use, please check all the screws are tighten and the main frame is stable. The maximum load weight is 100 kg.
2. Placed in ventilation place.
3. Verify that the test leads are in good condition. If necessary, replace them immediately if worn or missing.
4. Add the lubricating oil to the transmission parts regularly.

Cleaning








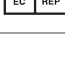
Use damp cloth with normal detergents. Do not submerge the seat in the bath water; drain after use and dry the seat. Don't use harsh abrasives or bleach based cleaners.

Working Condition

Temperature: 5 ÷ 38°C
 Humidity percentage: 20 ÷ 90% RH
 Atmospheric pressure: 800 ÷ 1060 hPa

Conservation condition and Transport


Temperature: - 10 ÷ 50°C
 Humidity percentage: 20 ÷ 90%
 Atmospheric pressure: 800 ÷ 1060 hPa

	Keep in a cool, dry place	REF	Product code
	Keep away from sunlight	LOT	Lot number
	Consult instructions for use	MD	Medical Device
	Medical Device compliant with Regulation (EU) 2017/745		Manufacturer
	Caution: read instructions (warnings) carefully		Date of manufacture
	Authorized representative in the European community	UDI	Unique Identifier

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

GIMA 27701 - 27702 - 27703

 **Foshan Dongfang Medical Equipment Manufactory (LTD.)**
 5 Xingfu Road, Zone A, Shishan Industrial Park,
 Nanhai District, Foshan City, China, 528000
 Made in China

REF FS893 - FS691 - FS691S

 **MedNet EC-Rep GmbH**
 Borkstrasse 10. 48163 Muenster. Germany

Importato da / Imported by:
Gima S.p.A.
 Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com
 export@gimaitaly.com
www.gimaitaly.com

