



ALZAWATER
RAISED TOILET SEAT
REHAUSSEUR WC
ALZAWATER
TOILETTENSITZERHÖHUNG
ΑΝΥΨΩΤΙΚΟ ΛΕΚΑΝΗΣ

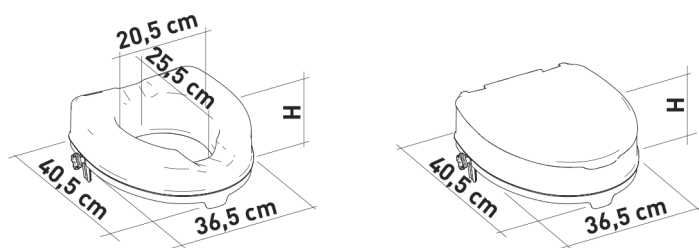

COD.	H	CON COPERCHIO - WITH LID AVEC COUVERCLE - CON TAPA MIT DECKEL - ΜΕ ΚΑΠΑΚΙ
27735	10CM/4"	
27737	15CM/6"	
27740	15CM/6"	
27741	10CM/4"	

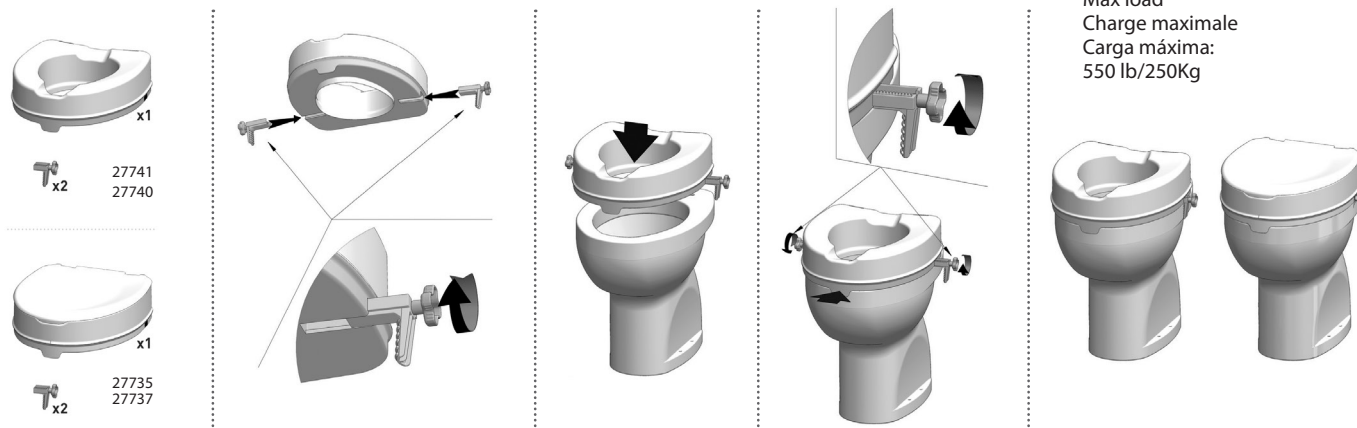

Carico massimo
 Max load
 Charge maximale
 Carga máxima:
 550 lb/250Kg

27735	ALZAWATER 10 CM CON COPERCHIO	RAISED TOILET SEAT 10 CM WITH LID	REHAUSSEUR WC 10 CM AVEC COUVERCLE	ALZAWATER 10 CM CON TAPA	TOILETTENSITZERHÖHUNG 10 CM MIT DECKEL	ΑΝΥΨΩΤΙΚΟ ΛΕΚΑΝΗΣ 10 CM ΜΕ ΚΑΠΑΚΙ
27737	ALZAWATER 15 CM CON COPERCHIO	RAISED TOILET SEAT 15 CM WITH LID	REHAUSSEUR WC 15 CM AVEC COUVERCLE	ALZAWATER 15 CM CON TAPA	TOILETTENSITZERHÖHUNG 15 CM MIT DECKEL	ΑΝΥΨΩΤΙΚΟ ΛΕΚΑΝΗΣ 15 CM ΜΕ ΚΑΠΑΚΙ
27740	ALZAWATER 15 CM	RAISED TOILET SEAT 15 CM	REHAUSSEUR WC 15 CM	ALZAWATER 15 CM	TOILETTENSITZERHÖHUNG 15 CM	ΑΝΥΨΩΤΙΚΟ ΛΕΚΑΝΗΣ 15 CM
27741	ALZAWATER 10 CM	RAISED TOILET SEAT 10 CM	REHAUSSEUR WC 10 CM	ALZAWATER 10 CM	TOILETTENSITZERHÖHUNG 10 CM	ΑΝΥΨΩΤΙΚΟ ΛΕΚΑΝΗΣ 10 CM

The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or non aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.





Alternatively, the RTS can be cleaned in domestic dishwashers at the maximum temperature.

REUSE REQUIREMENTS

The product is suitable for reuse by other users. Before reuse by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the normal household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose it.

REF	Product code	LOT	Lot number
	Medical Device compliant with Regulation (EU) 2017/745		Manufacturer
SN	Serial number		Date of manufacture
MD	Medical Device		Consult instructions for use
UDI	Unique device identifier		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

ENGLISH

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

INTENDED USAGE:

The products are designed to increase the height of the toilet bowl, in order to facilitate sitting down and getting up procedures and promoting independence and comfort. The versions provided with lid ensure better hygiene conditions during the toilet flush.

The lid can be used as a seat while in the closed position.

INDICATIONS AND CONTRAINDICATIONS:

The Raised Toilet Seat is aimed to elderly people, wheelchair users, people with reduced mobility of lumbar, knee or hip and to all those might have difficulty in sitting down and getting up from the toilet bowl. The ability to sit independently is required during use. It is not recommended to use the RTS in case of perception disorders or balance problems while sitting.

SAFETY WARNINGS AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the RTS is in a stable position on the WC and correctly fixed on it by the special fixing accessories supplied.
- Do not overtighten the locking system.
- During the assembly, strictly adhere what is shown in the User Manual.
- Do not modify the RTS and its components in any way.
- Do not use the RTS if damaged or altered in any way. Cracks and deformations may strongly affect the performance and safety of the device.
- Be sure the RTS height is such as to make the use of it easy and safe for the user.
- During use, avoid positions that keep the body weight too much forward. Wrong positions may lead to instability and overturning of the RTS.
- The RTS must not be stored outdoors..

MAINTENANCE INFO:

The products maintenance consists of the cleaning and sanitation procedures. In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

REF

27735 - 27737 - 27740 - 27741



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
Made in Italy

