# **GIMA**

PROFESSIONAL MEDICAL PRODUCTS

---

## ***BASTONE ALLUMINIO REGOLABILE ADJUSTABLE ALUMINIUM WALKING STICK***

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**All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.**

---

## INTRODUCTION

This walking stick can be adjusted in height. Made of aluminium which is light and resistant. The tip is covered with non-slip rubber.

Adjustable Height

COD. 27780 (from 71 to 94 cm)

COD. 27781 (from 67 to 90 cm)

COD. 27784 (from 77 to 100 cm): the length must be adjusted according to the height of the user.

## SAFETY RULES

Always contact your specialist dealer to adjust the device and to learn how to use it appropriately.

Do not store outdoors, do not expose it to prolonged periods of sunlight. Before using it check that all connections are tightened correctly.

Always move slowly, taking small steps to ensure that the weight distribution is even and that the position is stable and safe.

Do not use on slippery surfaces (wet floors, snow, ice) or on stairs.

Do not use for purposes other than those for which this product has been designed.

Always comply with the weight limits indicated on the label or in the instructions.

Use only original spare parts for repairs. Only qualified personnel are authorised to repair this item. Should you not use the device for long periods of time, have it checked by your specialist dealer.

Use caution when using on unstable and uneven ground. Check the wear of the non-slip rubber and replace it if necessary.

## INSTALLATION

The commissioning of this product should only be carried out by qualified personnel, incorrect adjustment could result in an accident.

Height adjustment: loosen the clamping ring. Press the button and slide the tube up to the desired adjustment hole.

Return the button to the starting position. Screw in the clamping ring.

To fold it, unhook and loosen the pipes.

## MAINTENANCE

The product must be cleaned regularly with a clean and dry cloth. The handle must be cleaned with a neutral detergent (do not use corrosive products). The ends must be checked and changed regularly in case of wear.

## GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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## GIMA 27780 - 27781 - 27784

**Foshan Dongfang Medical Equipment Manufactory (LTD.)**

5 Xingfu Road, Zone A, Shishan Industrial Park,

Nanhai District, Foshan City, China, 528000

Made in China

**FS920L - FS939L - FS938L(12)**

**MedNet EC-Rep GmbH**

Borkstrasse 10. 48163 Muenster.

Germany

Importato da / Imported by:

**Gima S.p.A.**

Via Marconi, 1 - 20060 Gessate (MI) Italy

[gima@gimaitaly.com](mailto:gima@gimaitaly.com) - [export@gimaitaly.com](mailto:export@gimaitaly.com)

[www.gimaitaly.com](http://www.gimaitaly.com)