PROFESSIONAL MEDICAL PRODUCTS

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# **FOLDING ALUMINIUM WALKING STICK**

# **BÂTON PLIABLE EN ALUMINIUM**

# **PALO DE ALUMINIO PLEGABLE**

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**All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located**

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**REF FS927L (GIMA27782)**

Foshan Dongfang Medical Equipment Manufactory (LTD.)  
5 Xingfu Road, Zone A, Shishan Industrial Park,  
Nanhai District, Foshan City, China, 528000  
Made in China

MedNet EC-Rep GmbH  
Borkstrasse 10. 48163 Muenster.  
Germany

Importato da / Imported by /  
Importé par / Importado por:  
Gima S.p.A. Via Marconi, 1  
20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

## INTRODUCTION

This walking stick can be folded into 4 sections and may be stored in a bag when no longer in use. Made of aluminium which is light and resistant. The tip is covered with non-slip rubber. Adjustable Height (from 80 cm to 90 cm.): the length must be adjusted according to the height of the user.

## SAFETY RULES

Always contact your specialist dealer to adjust the device and to learn how to use it appropriately. Do not store outdoors, do not expose it to prolonged periods of sunlight. Before using it check that all connections are tightened correctly.

Always move slowly, taking small steps to ensure that the weight distribution is even and that the position is stable and safe.

Do not use on slippery surfaces (wet floors, snow, ice) or on stairs.

Do not use for purposes other than those for which this product has been designed.

### Max load 100 Kg.

Use only original spare parts for repairs. Only qualified personnel are authorised to repair this item.

Use caution when using on unstable and uneven ground. Check the wear of the non-slip rubber and replace it if necessary.

## INSTALLATION

The commissioning of this product should only be carried out by qualified personnel, incorrect adjustment could result in an accident.

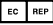
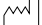
Height adjustment: loosen the clamping ring. Press the button and slide the tube up to the desired adjustment hole.

Return the button to the starting position. Screw in the clamping ring.

To fold it, unhook and loosen the pipes.

## MAINTENANCE

The product must be cleaned regularly with a clean and dry cloth. The handle must be cleaned with a neutral detergent (do not use corrosive products). The ends must be checked and changed regularly in case of wear

Symbols					
	Medical Device compliant with Regulation (EU) 2017/745		Manufacturer		Product code
	Caution: read instructions (warnings) carefully		Keep away from sunlight		Lot number
	Consult instructions for use		Keep in a cool, dry place		Medical Device
	Authorized representative in the European community		Date of manufacture		

## GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies