## SINUS A

Nasal Wash

120 ml Bottle + 12 Saline Sachets

# SINUS A

**Composition:** Sodium Chloride, Sodium Bicarbonate

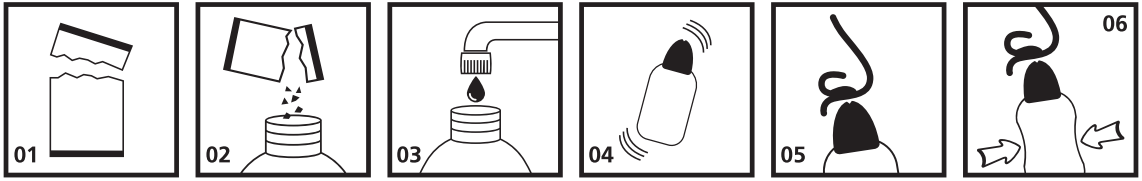
**Properties:** Nasal washing alleviates symptoms caused by allergies, sinus infection, dry air, nasal congestion, Post-nasal drip and rhinitis of pregnancy. It has been clinically proven to provide effective relief for uncomfortable sinus and nasal conditions and promote better breathing. The bottle uses saline solution to wash the nasal passages. It's all natural and is safe for children up to 12 years old and adults, including pregnant and nursing mothers and people taking other medications. Simply dissolve a Sachet of the pre-mixed saline ingredients in warm water inside the bottle, shake it well, and squeeze the bottle and refresh and protect the nasal passages.

**Indications:** Helps in relieving sinus symptoms, post nasal drip, congestion, dryness, nasal allergies, nasal irritation from occupational dust, fumes, animal dander, grass, pollen, smoke, smog and house dust.

### DO NOT USE BELOW 12 YEARS OLD!

**Direction of use:** Put 1 Sachet inside the bottle (02). Fill it with water until 120 ml and shake it well (03 & 04). Place the nozzle in each nostril (05). Squeeze the bottle (06). Repeat this step until you feel the flow of the solution through the other nostril, repeat the previous two steps to decongest the nose.

See the pictures below.

**Dosage:** Adults & Children 12 years & over: 1-2 sachets up to every 2 hours as often as needed.

**Warning:** Stop use and ask a doctor if washing is uncomfortable or symptoms are not relieved. This product is for personal use, do not use same bottle for more than one person. Wash with soap and water after each use.

**Side Effects:** There have been relatively few side effects reported with the proper use of sinus nasal wash, and they are considered minor. These include itching and slight stinging sensation inside the nose. Discontinue use and consult your doctor if you develop an allergic reaction to sinus nasal wash.

Sinus Nasal Wash is safe to be used during Pregnancy & Breast-feeding.

**Storage:** Protect Saline Sachets from excessive heat and moisture, between 5-30°C.

Use within 12 months of opening.

**Keep out of reach of children.**

**Packaging:** 120 ml Bottle + 12 Saline Sachets (3g).

In case of accidental eye exposure, immediately and thoroughly flush with fresh water, also under eyelids, for at least 15 minutes. Seek for medical attention if irritation persists.

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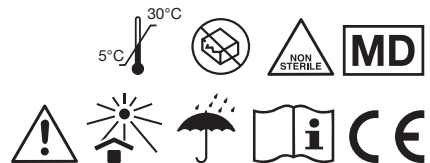
All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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**Gima S.p.A.** Via Marconi, 1 - 20060 Gessate (MI) Italy