HOSPYNEB PROFESSIONAL

POMPA SINGOLA

SINGLE PUMP

POMPE SIMPLE

EINZELPUMPE

BOMBA SIMPLE

GUARANTEED

3

YEARS

HOSPITAL

- 1 – Interruttore ON/OFF / ON/OFF Switch / Interrupteur ON/OFF / Ein-Ausschalter (ON/OFF) / Interruptor ON/OFF
- 2 – Regolatore pressione all'ampolla / Operating pressure regulator / Régulateur de pression à l'ampoule / Druckregler der Sprühflasche / Regulador de presión a la ampolla
- 3 – Manometro visualizzazione pressione all'ampolla / Operating pressure gauge / Manomètre d'affichage pression à l'ampoule / Manometeranzeige des Sprühflaschendrucks / Manómetro visualización de presión a la ampolla
- 4 – Connettore aria / Outlet air connector / Connecteur air / Luftanschluss / Conector de aire
- 5 – Filtro aria in aspirazione / Air filter / Filtre à air d'aspiration / Saugluftfilter / Filtro de aire de aspiración

ACCESSORI IN DOTAZIONE / ACCESSORIES INCLUDED / ACCESSOIRES STANDARD / AUSSTATTUNGSZUBEHÖR / ACCESORIOS ESTÁNDAR:

- Kit nebulizzatore FASTERJET (Nebulizzatore - Maschere Pediatrica e Adulti Boccheruola – Forcella Nasale)
VEDI MANUALE ACCESSORI
- FASTERJET nebulizer kit (Nebulizer - Pediatric and Adult Masks Mouthpiece - Nosepiece)
SEE ACCESSORIES MANUAL
- Kit nébulisateur FASTERJET (Nébulisateur - Masques enfant et adulte Embout buccal - Fourche nasale)
VOIR MANUEL ACCESSOIRES
- Bausatz Zerstäuber FASTERJET (Zerstäuber - Maske für Kinder und Erwachsene - Mundstück- Nasenstück)
SIEHE GEBRAUCHSANLEITUNGEN DES ZUBEHÖRS
- Kit nebulizador FASTERJET (Nebulizador - Mascarillas pediátrica y adulto Boquilla - Horquilla nasal)
VÉASE MANUAL DE ACCESORIOS
- Ricambio filtro porex
- Porex filter replacement
- Filtre de rechange porex
- Ersatzfilter Porex
- Repuesto filtro porex

Medical Device (Class IIa – MDD 93/42/EEC) – Read the instructions carefully before use

HOSPYNEB is a professional device for the administration of medication for the treatment and relief of the respiratory tract intended for use to administer aerosoltherapy on the recommendation of a physician.

HOSPYNEB is equipped with a piston compressor with incorporated air filter that is periodically replaced (every 70 applications). Reliable, durable and lubrication-free, it is constructed in observance of current European regulations for the safety of devices for electromedical use (EN 60601-1; EN13544-1) and has been tested for conformity to EMC standards (EN60601-1-2);

HOSPYNEB includes accessories manufactured by 3A (see accessories manual) that have been laboratory tested in conformity with the most recent scientific criteria for the effectiveness and biological safety (the bio-compatibility of the accessories is certified by specialized analysis laboratories).

NOTICES

This is a medical device for home/professional use and must only be used with a doctor's prescription. It must be operated as indicated in this instruction manual. It is important that the patient read and understand the information for the use and maintenance of the unit. Contact a trusted reseller for any questions.

Do not use this device for any purpose other than its intended use, i.e., as an aerosoltherapy device. The manufacturer is not responsible for improper use.

The device is not suitable for anesthesia or pulmonary ventilation.

This equipment is not suitable for use in the presence of an inflammable anesthetic mixed with air, oxygen or nitrous oxide.

The device's cover is not protected against the penetration of liquids.

The device must only be used with its original accessories, as shown in the instruction manual

Always use the device and its accessories following the recommendations of your doctor. Use only medications prescribed by your doctor and only in the manner that the latter has indicated.

Do not use extension cords or adapters and we recommend always unwinding the electric cord to avoid dangerous overheating.

Keep the device and its cord away from hot surfaces. Always unplug the cord after use.

Never immerse the product in water; if this should occur, immediately disconnect the plug from the electric socket. Do not touch or attempt to remove the submerged device from the water without having first disconnected the plug from the electric socket.

Do not use the device after removing from the water (immediately send it to a reseller you trust).

Do not open and/or tamper with the device. If the device is not working correctly, turn it off and consult the instruction manual. Contact a trusted reseller for any repairs.

The device contains small parts that can be removed and easily swallowed. Its use by minors or the disabled thus requires the presence of an adult with full mental faculties. Do not leave the device unattended in places easily accessible by minors and the disabled.

Keep in a clean, dry place, protected from direct sunlight, sources of heat and the weather.

Dispose of the device in conformity with current law.

fig. A

fig. B

fig. C

INSTRUCTIONS FOR PUTTING THE DEVICE INTO OPERATION

1. Insert the plug into an electric socket with the correct voltage for the device.

2. Unscrew the ampoule bottle so that it separates into two parts (fig. A). 3. Fill the lower part of the ampoule with the medication prescribed by the doctor (fig. B). Notice: the level notches in the lower part of the ampoule have an

indicative value only. 4. Screw the two parts of the ampoule back together (fig. C). 5. Insert the tube coming from the compressor OUTLET (fig. C). 6. Using the specific connector provided, attach the selected treatment accessory: mask, mouthpiece or nosepiece. 7. When the application is completed, turn-off the device and remove the plug from the electric outlet.

Fig. 1a

Fig. 1b

CLEANING AND DISINFECTION

Carefully wash your hands before cleaning and disinfecting the accessories.

Replacing the filter: Lift the filter cover and use a flat screwdriver as a lever, as shown in figure 1a; detach the filter to be replaced and insert the replacement, as shown in figure 1b. Finally, insert the filter cover in its seat, making it adhere to the surface. Compressor cleaning: Never immerse the device or wash it under

water; if you wish to clean it, use only a damp cloth with a (non-abrasive) detergent. Accessory cleaning: See the accessory instruction manual

PROBLEMS, CAUSES AND SOLUTIONS

THE DEVICE WON'T TURN-ON

- Make sure the plug is fully-inserted in the electric outlet.
- Make sure the switch is in the on position (I).

THE DEVICE IS ON BUT DOESN'T NEBULIZE

- Make sure you have inserted the nebulization nozzle (pisper) inside the ampoule.
- Make sure the air connection tube is not crushed or bent.
- Make sure the aspiration filter is not obstructed or dirty. If so, replace it with a new one.

- Make sure that there is medication in the ampoule.

THE DEVICE DOESN'T WORK

- The thermoprotector has been tripped for the one of the following reasons:
 - the device has operated near a source of heat or in an environment with temperatures exceeding 40 °C.

Let the device cool for at least 60 minutes and then turn it on again.

N.B.: In the event the device does not resume working despite having made the above checks, contact a trusted reseller.

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