


**HEEL PROTECTOR**

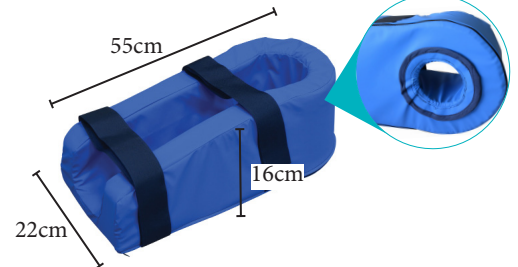

Ref. 680 011      Αναφ. 680 011  
45\*10\*15 cm 150gr 45\*10\*15 εκ. 150 γρ.

Ref. 680 008      Αναφ. 680 008  
22\*15.12 cm 90gr 22 \* 15,12 εκ. 90 γρ

Ref. 680 010      Αναφ. 680 010  
22\*15.12 cm 90gr 22 \* 15,12 εκ. 90 γρ



	FR	EN	DE	ES	
	Signification des symboles	Explanation of symbols	Zeichenerklärung	Explicación de los símbolos	
1	Dispositif Medical	Medical Device	Medizinprodukt	Dispositivo médico	
2	Lire le mode d'emploi avant l'utilisation.	Read the instruction manual before use	Vor Gebrauch Gebrauchsanweisung lesen!	Lea el manual de instrucciones antes de utilizarlo	
3	Respecter les instructions de sécurité!	Observe the safety instructions!	Sicherheitshinweise beachten!	Siga las instrucciones de seguridad	
4	Conformité CE	CE conformity	CE-Konformität	Conformidad con la normativa CE	
	PT	PL	HU	TR	
	Significado dos símbolos	Objaśnienia symboli	Szimbólumok jelentése	Sembollerin açıklamaları	
1	Dispositivo médico	Wyrób medyczny	Orvosi eszköz	Tıbbi cihaz	
2	Antes de utilizar, leia, por favor, o manual.	Przed użyciem zapoznaj się z instrukcją obsługi	Használat előtt olvassa el a használati útmutatót	Kullanmadan önce kullanma kılavuzunu okuyunuz	
3	Siga as instruções de segurança	Stosuj się do instrukcji bezpieczeństwa.	Tartsa be a biztonsági utasításokat!	Güvenlik uyarılarını dikkate alınız	
4	Certificado CE	Deklaracja CE	CE megfeleléség	CE uygunluğu	
	IT	NL	SI	HR	GR
	Significato dei simboli	Verklaring van de symbolen	Pomen simbolov	Objašnjenje simbola	Επεξήγηση Συμβόλων
1	Dispositivo Medico	Medisch hulpmiddel	Medicinski pripomoček	Medicinski uređaj	Ιατρική Συσκευή
2	Prima dell'uso leggere le relative istruzioni!	Voor gebruik de handleiding lezen.	Pred uporabo preberite navodila	Pročitati korisničke upute prije uporabe	Διαβάστε τις οδηγίες πριν από τη χρήση
3	Osservare le istruzioni per la sicurezza	Veiligheidsinstructies respecteren	Upoštevajte varnostna navodila!	Pažljivo pogledajte sigurnosne upute	Τηρείτε τις οδηγίες ασφαλείας!
4	CE conformità	CE conformiteit	CE Certifikat	CE-Deklaracija	CE Συμμόρφωση

**EN-** Please read these safety instructions carefully before use, and keep them for future question.

**1. Safety indications**

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store heel support outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!), store in a dry room in a flat position to prevent deforming the cushion.
- ♥ Store the heel support away from sharp objects and pointed edges to prevent tears and punctures of the cushion and cover.
- ♥ Use and store cushion in a flat unfolded position.
- ♥ Look for cracks before use.
- ♥ Don't use in extreme temperature conditions (above 38°C or 100°F or below 0°C or 32°F).
- ♥ No modification must be made to this device.
- ♥ ALWAYS check if the protector is properly and securely locked in place before using.
- ♥ In case you didn't use your cushion for a long time, let it be checked by the technical staff, before using.
- ♥ ALWAYS observe the weight limit on the labelling of your device.
- ♥ Use only as a protector and in a proper way: don't knee or step on the product.
- ♥ If your skin develops redness, discontinue the use of the cushion immediately and consult your doctor or therapist.
- ♥ Follow the advice of the professional who prescribed this medical device, respect duration and frequency of use.
- ♥ In case of discomfort consult your professional.
- ♥ Avoid contact with any sharp objects or pointed edges that may tear or puncture the cushion or the cover.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

**2. Intended use, indications and contraindications**

It is easier to prevent pressure ulcers than to treat them. Bedsores can be avoided by changing frequently lying or seating positions to avoid stress on

the skin and to minimize the risk of pressure on vulnerable bony areas. Other strategies include skin care, regular skin inspections and good nutrition. Using a heel protector can help relieve pressure and make sure that the heel is correctly positioned in the chair or in the bed. It is one of the best ways to avoid bedsores, specifically reducing pressure, friction and shear from the heels, „floating“ the heels off the surface of the bed or chair.

**Which patients are at risk of pressure ulcers?**

Patients with limited or no mobility due to their age, an accident or illness, and who sit for long periods of time in the day. Patients who cannot reposition themselves independently when they get uncomfortable. These patients might slump to one side, forward or slide down the chair, needing help from carers or family members to regain a mid-line posture. Patients who have thin or fragile skin and who are at a high pressure injury risk. Patients who are hoisted for all transfers.

Indications:

- Patients with or at risk of developing minor pressure ulcers
- Pain relief, pressure redistribution, reduction of friction and shear forces
- Patients with significant bony prominences
- Post surgery and bedsores prevention.

Contraindications:

- Don't use this product in case of :
- Excessive sweating causing a risk of skin maceration
  - Severe skin defect (ulcers)
  - Inadequate supporting surface
  - Avoid layering incontinence products

**3. Description and Fitting instructions**

Ref 680010

The heel protector is a contoured moulded visco-elastic foam support. The foam increases the contact surface and maximizes pressure relief. The heel protector's shape ensures a good posture of the leg and completely offloads heels. This heel protector is delivered with a polyurethane cover.

Hook and loop closure.

Ref 680011

Coated cylindrical hollow core fibres provide soft comfortable support, while

still keeping the rest of the foot cool. Two soft elastic velcro straps help keep the heel protector in place.

Ref 680008

Our synthetic fur heel protectors are recommended for patients at low risk for skin breakdown and friction burns, and offer comfort and warmth against the skin while still allowing air to circulate and feet to breathe. Easy hook and loop closure.

#### **4. Washing instructions**

Ref 680010

To clean Cover:

Remove the cover from the cushion. Cover is machine washable at 30°C. Dry flat and away from any source of heat. Do not use detergent or bleach, do not autoclave.

To clean Cushion:

Remove the cover, then wipe lightly with a damp cloth. Do not use soap and do not submerge in water. After cleaning make sure to wipe dry and air dry the heel protector before using. Re-install the cover over the heel protector.

Ref 680011

Machine washable cold. Dry flat and away from any source of heat. Do not use detergent or bleach, do not autoclave.

Ref 680008

Machine washable at 40°C. Dry flat and away from any source of heat. Do not use detergent or bleach, do not autoclave.

#### **5. Composition**

Ref 680010

Polyurethane viscoelastic foam developed by BASF. Cover is made of polyurethane.

Ref 680011

Silicone hollow core fibers inside. Cover is made of polyurethane.

Ref 680008

100% Polyester

#### **6. Disposal**

Dispose of this product in accordance with your regional waste disposal regulations.

#### **7. Technical specifications**

See above information. This product has a 2-year warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible. This product is CE certified. Duration of use: 2 years.