



Hot water bottle double ribbed



Hot water bottle double ribbed

Instructions for use

Follow the indications below to make your bottle last longer. Do not use hot boiling water and fill bottle only up to 1/3 of its capacity. Before closing with the cap let steam out, Dry bottle and keep away from heat

Guidelines for the safe use of the hot water bottle

Fill with water at temperature that does not exceed 45 degrees - Avoid compressing the hot water bottle during use - The temperature at which the hot water bottle is stored should be between 10 and 27 degrees Celsius - Always check the general state of the rubber, correct screwing of the cap and overall sealing before use. Avoid keeping the bag in direct contact with the skin, but rather wrap it in a cloth. All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

MD

Dispositivo medico
Medical Device
Dispositif medical
Medizinprodukt
Izopornovokoyed robitel
Producto sanitario
Dispositivo medico

Sadržaj
Energy
Date of issue
Frist
Recycle
Fecha tope
Prato de entrega



28600-M-Rev.1.10.21

REF **28600**

LOT **000013**



Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) - Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
Made in China



50849
Size 22x44.5 cm