
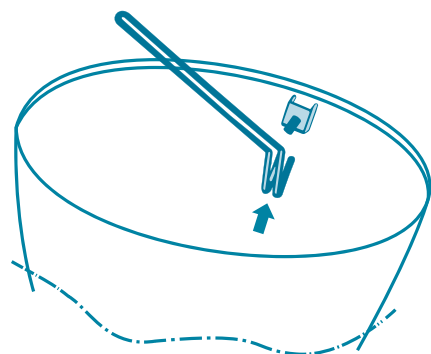


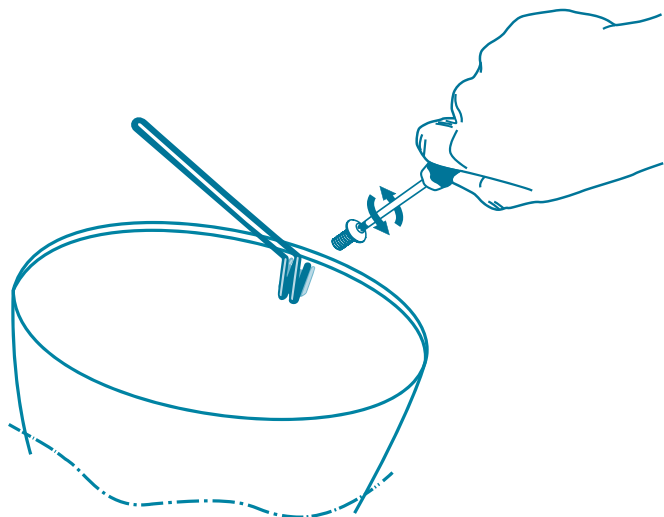

INFRA 150 PLUS

IMPORTANTE.-

Uso y tiempo de aplicación bajo supervisión y responsabilidad médica.

IMPORTANT.-

Use and time of application under medical responsibility and supervision.

IMPORTANT.-

L'utilisation et le temps d'application sous surveillance et responsabilité médicale.

IMPORTANTE.-

Uso e tempi di applicazione sotto la supervisione e la responsabilità medica.

IMPORTANTE.-

Usar e tempo de aplicação sob supervisão e responsabilidade médica.

MIMSAL

MIMSAL TRADE S.L.

C. Mollet 17, Pol. Ind. Palou Nord - 08401 Granollers (Barcelona) - Spain
Tel. +34 930 139 860 - mimsal@mimsal.com - www.mimsal.com

INFRA PLUS



MIMSAL

INFRA 150 PLUS

INFRA 250 PLUS

cod. E020412

cod. I0103512

Infra-red lamp for therapeutic use in beauty treatment, physiotherapy and in the sports area.

Heat is distributed evenly.

1. Installation

- Requires a MIMSAL mount, which is installed before the lamp is placed in position.
- A table clamp mount is supplied and wall or ceiling mounts or a rotating base can be supplied as optional accessories.
- MIMSAL cannot guarantee the life and functionality of the product where third party accessories are used.

2. Precautions

- Always disconnect the lamp from the power supply when cleaning or carrying out maintenance work.
- Clean either dry or using a weak solution of normal soap and water. Avoid using alcohol, acids or any type of organic solvent.
- Do not leave on for longer than 8 hours at a time.

3. Troubleshooting and maintenance

- **Maintenance:** Only for qualified electrical engineers or other qualified personnel.
- **To change the bulb:**
 - Always disconnect the lamp from the power.
 - Replace the bulb.

Please contact your MIMSAL rep if you have any other problem.

MIMSAL only accepts responsibility for maintenance, repair or changes carried out by MIMSAL staff using original MIMSAL parts.

The manufacturer reserves the right to change the characteristics of its products without prior notice.

	INFRA 150 PLUS	INFRA 250 PLUS
Light bulb	IR E-27 150W	IR E-27 250W
Head diameter	21 cm	30 cm
Fitting	E-27 Ceramic	E-27 Ceramic
Length of Arm	105 cms (50+55cm)	105 cms (50+55cm)
Electrical Protection Class	Class II	Class II
Color	White	White
Power	220/230V	220/230V