



# GIMA

PROFESSIONAL MEDICAL PRODUCTS

**SCALDALETTO ELETTRICO**  
**HEATING UNDERBLANKET**  
**CHAUFFE-LIT**  
**AQUECEDOR DE COLCHÃO**  
**VÄRMEUNDERFILT**  
**VUOTEENLÄMMITIN**  
**WÄRMEUNTERBERR**  
**SALTEA ELECTRICĂ**  
**ΘΕΡΜΑΙΝΟΜΕΝΟ ΥΠΟΣΤΡΩΜΑ**  
**VYHŘÍVANÁ PODLOŽKA**  
**PODGRZEWANY PODKŁAD**  
**ELEKTRISCHE ONDERDEKEN**

**REF** SS29(150\*080)-1X (GIMA 28660)  
SS29(160\*140)-2X (GIMA 28661)

Fabbricante / Manufacturer / Fabricant / Fabricante / Hersteller / Κατασκευαστής  
/ Producent / Výrobce / Producent / Producent / Tillverkare / Valmistaja  
**Dongguan Sunbright Electric Appliance Co., Ltd.**  
No.102, Shangdong Yiheng East Road, Qishi Town,  
523455 Dongguan City, Guangdong Province, China  
Made in China

Importato da / Imported by / Importé par / Importado por / Importiert von / Εισάγεται από  
/ Importowane przez / Importováno / Importat de / Importerad av / Geïmporteerd door /  
Maahantuoja:  
**Gima S.p.A.**  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)





Class II applied



Product complies with European Directive



Follow instructions for use



Product code



Caution: read instructions (warnings) carefully



Lot number



WEEE disposal



	Not suitable for children under 3 years
	Do not puncture the heated cushion!
	Do not use folded or rucked
	Do not tumble dry
	Do not dry-clean
	Do not iron
	Do not bleach
	Machine wash

ENGLISH

## Instruction Manual

### Electric blankets

SS29(150\*080)-1X  
SS29(160\*140)-2X

Please read this instruction manual carefully before use and keep it for future reference.

**IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!**

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

#### IMPORTANT SAFETY INSTRUCTION

- Check the blanket carefully before each use.
- If you notice visible damage on the blanket, do not use and return to the manufacturer or dealer.
- The blanket can only be operated with the controller.
- Before connecting the blanket to the power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- The blanket may not be operated while folded, creased, wrapped in a cover or placed around the mattress.
- Four fastening straps are located on the corner of the blanket. Use these straps to attach the blanket to the mattress using a suitable band or cord. Make sure that the blanket is firmly strapped to the mattress in both directions and that it will not crease while it is being

used. Check if it is in the correct position before lying on it.

- During use, the controller switch and cable must not be placed on or under the blanket or be covered in any way.
- Before using it on an adjustable bed, make sure that the power cable and the blanket do not get caught in the hinges or get crumpled when making adjustments.
- Neither pierce the blanket with, nor attach safety pins or other spiky or sharp objects to the pad.
- Particularly do not use the electric under blanket with the infirm, infants or persons insensitive to heat.
- Not to be used by very young children (0-3 years). Children under the age of three are not to use this appliance due to their ability to react to overheating.
- Children shall not play with the blanket.
- Cleaning and user maintenance shall not be made by children without supervision.
- Never use the blanket unsupervised.
- Prolonged use of the blanket at a high setting could lead to skin burns.
- Prolonged use of the blanket at a high setting could lead to skin burns.
- Never touch a blanket that has fallen into water. Unplug the blanket from the mains outlet immediately.
- Keep the mains cable away from hot surfaces.
- Never carry, pull or turn the blanket by the mains cable and do not let the cable get tangled.
- Do not use the blanket when wet and only use it in dry environments (not in the bathroom or similar environments).
- The controller and cable must not be exposed to any kind of moisture.
- If a fault occurs, do not attempt to repair the blanket by yourself. Repairs must only be carried out by an authorized specialist dealer or other suitably qualified personnel.
- If the power supply cable is damaged, it may only be replaced by, an authorised dealer or by qualified electricians. In order to prevent hazards, always send the blanket in to the service centre.
- Let the blanket cool down completely before folding and storing it. Do not fold it too tight.
- When not in use, store the blanket as follows :
  - Store the blanket in its original packaging and keep it in a clean, dry place.
  - Fold the power cable carefully to avoid cable breaks.
  - Do not crease the blanket by placing items on top of it during storage.
- Should you have any concerns about health issues, consult your doctor before using the blanket.
- If you experience any pain or discomfort while using the blanket, stop using it immediately.
- The blanket is only for private use and is not to be used in hospitals.

#### OPERATION

It is recommended that the blanket to be switched on (in position 3) 30 minutes before going to bed. Plug the blanket into the mains outlet and turn the selector switch on the controller from position 0 to position 1. The red power indicator will light up. You will feel the blanket gradually warming up. This will be even more noticeable if it is covered with a quilt. If you require a higher temperature setting, turn the switch to the position 2 or to position 3. Should the blanket get too warm for you, turn the switch back to position 2 or to position 1. Position 2 and 3 of the blanket is intended for heating up, only position 1 is for continuous operation. Move the selector switch to position 1 before falling asleep. To switch off the blanket, turn the switch to position 0. The red power indicator will go off, indicating that the unit is switched off. Now unplug the blanket from the mains outlet.

#### CLEANING AND MAINTENANCE

- Before cleaning the blanket, unplug it from the mains outlet and let it cool down completely or for at least ten minutes.
- The blanket is equipped with a detachable cable.

Disconnect the plug and remove the cable from the blanket.

- You may wash the blanket gently by hand. The best way to do this is to put the blanket in a bath with lukewarm water and some mild detergent and then gently squeeze it out.
- Do not use hair dryer to dry this product, have to dry naturally, stretch the blanket out on a flat surface.
- Rinse the blanket several times to remove all the detergent.
- Store the blanket in a clean and dry place, laid out flat with nothing covering or resting on top of it.
- Only use the blanket once it is completely dry.
- You may also wash the blanket in a washing machine at a maximum temperature of 30°C with low mechanical action and spin. It's advisable to wash it by hand in order to prolong the life of the product.

#### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies



**Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment