



# GIMA

PROFESSIONAL MEDICAL PRODUCTS

**CINTURA RISCALDANTE**

**WAIST WRAP HEATING PAD**

**CEINTURE CHAUFFANTE**

**ALMOFADA D AQUECIMENTO D ENROLAR À  
CINTURA**

**HEIZWICKELKISSEN FÜR DIE HÜFTE**

**ΘΕΡΜΟΦΟΡΑ ΜΕΣΗΣ**

**PAS LĘDŹWIOWY ELEKTRYCZNY  
ROZGRZEWAJĄCY**

**ZAHŘÍVACÍ BEDERNÍ PÁS**

**SĂCULEȚ ÎNCĂLZITOR CU CUREA PE TALIE**

**VÄRMEDYNA MED MIDJEBAND**

**WARMTEGORDEL BUIK EN RUG**

**VYÖTÄRÖLLE KIEDOTTAVA LÄMPÖTYYNY**

**REF** SS02A(29\*69) (GIMA 28675)

Fabbricante / Manufacturer / Fabricant / Fabricante / Hersteller / Κατασκευαστής  
/ Producent / Výrobce / Producent / Producent / Tillverkare / Valmistaja

**Dongguan Sunbright Electric Appliance Co., Ltd.**

No.102, Shangdong Yiheng East Road, Qishi Town,  
523455 Dongguan City, Guangdong Province, China

Made in China

Importato da / Imported by / Importé par / Importado por / Importiert von / Εισάγεται από  
/ Importowane przez / Importováno / Importat de / Importerad av / Geïmporteerd door /  
Maahantuojat:

**Gima S.p.A.**

Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com

[www.gimaitaly.com](http://www.gimaitaly.com)



## Instruction Manual SS02A(29\*69)

**IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!**  
Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.  
Should you give this device to another person, it is vital that you also pass on these instructions for use.

This instruction manual belongs to this device.  
It contains important information about starting up and operation.  
Read the instruction manual thoroughly.  
Non-observance of these instructions can result in serious injury or damage to the device



### WARNING

These warning notes must be observed to prevent any injury to the user.



### CAUTION

These notes must be observed to prevent any damage to the device.



### NOTE

These notes give you useful additional information on the installation or operation.



Class II applied



Product complies with European Directive



Follow instructions for use



Product code



Caution: read instructions (warnings) carefully



Lot number



WEEE disposal



Do not puncture the heated cushion!



Do not use the heated cushion when it is folded!



Not to be used by very young children (0-3 years)!



Only use the heated cushion indoors!



The heating pad can be washed by hand! You may wash the cushion cover in a normal wash at max. 30 °C!



Do not chlorinate!



Do not dry the heated cushion in a tumble drier!

The cushion cover may not be put in a dryer!



Do not iron the heated cushion!

You may iron the cushion cover at a low temperature!



Do not dry clean!

### SAFETY INFORMATION

- Carefully check the heat pad before each use, if you notice wear or damage.
- Do not use if you notice wear, damage or signs of improper use on the device, the switch or the cables or if the device is not working.
- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Do not fold or crease the heat pad during use.
- Neither pierce the pad with, nor attach safety pins or other spiky or sharp objects to the pad.
- Do not use the device on children or persons who are disabled, asleep or insensitive to heat (persons who cannot react on overheating).
- Children below the age of 3 years are not allowed to use this device, because they are not able to react on overheating.
- The device must not be used by young children over the age of 3 years, unless the control unit has been adjusted accordingly by its parents or legal guardians or unless the child has been well instructed how to use the control unit properly.

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision.
- Never use the pad unsupervised.
- The device is only for private use and is not to be used in hospitals.
- Do not fall asleep while the heat pad is switched on.
- Prolonged use of the heated cushion at a high setting could lead to skin burns.
- Do not sit on the heat pad. Instead, rest the pad on or against the part of the body to be treated. The intended use of the is to treat only back and neck.
- Never touch a heat pad that has fallen into water. Unplug the unit from the mains outlet immediately.
- Keep the mains lead away from hot surfaces.
- Never carry, pull or turn the heat pad by the mains lead and do not let the lead get tangled.
- Do not use the heat pad when wet and only use it in dry environments (not in the bathroom or similar environments).
- The controls and leads must not be exposed to any kind of moisture.
- If a fault occurs, do not attempt to repair the unit yourself. Repairs must only be carried out by an authorized specialist dealer or other suitably qualified personnel.
- If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
- A damaged power supply cable must be replaced with a power cable of the same type.
- Heat pad may only be operated with the control unit (SS02) supplied.
- When storing the heat pad, allow it to cool down before folding.
- Should you have any concerns about health issues, consult your doctor before using the heat pad.
- Do not use the heat pad on parts of the body that are swollen, injured or inflamed.
- Should you experience persistent pains in the muscles and joints, please inform your doctor. Persistent pains can be a symptom of serious illness.
- If you experience any pain or discomfort while using the unit, stop using it immediately.

## CLEANING AND MAINTENANCE

### Heat pad

- Before cleaning the pad, unplug it from the mains outlet and let it cool down for at least ten minutes.
- The heating pad is equipped with a detachable cable. Disconnect the plug and remove the cable from the heating pad.
- You may wash the heating pad gently by hand. The best way to do this is to put the heating pad in a bath with lukewarm water and some mild detergent and then gently squeeze it out.
- Rinse the heating pad several times to remove all the detergent.
- Store the unit in a clean and dry place, laid out flat with nothing covering or resting on top of it.
- Only use the appliance once it is completely dry

## GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies



**Disposal:** *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.*