GIMA

PROFESSIONAL MEDICAL PRODUCTS

SACCA STERILE PER IL DRENAGGIO DELLE URINE PER USO SINGOLO
STERILE URINE DRAINAGE BAG FOR SINGLE USE
SAC DE DRAINAGE D'URINE STÉRILE À USAGE UNIQUE
STERILER URIN-DRAINAGEBEUTEL FÜR DEN EINMALIGEN GEBRAUCH
BOLSA DE DRENAJE DE ORINA ESTÉRIL DE UN SOLO USO
SACO DE DRENAGEM DE URINA, ESTÉRIL PARA UTILIZAÇÃO ÚNICA
ΑΠΟΣΤΕΙΡΩΜΕΝΟΣ ΟΥΡΟΣΥΛΛΕΚΤΗΣ ΜΙΑΣ ΧΡΗΣΗΣ
STERILNÍ SÁČEK NA ODVOD MOČI NA JEDNO POUŽITÍ
STERIILNE ÜHEKORDELT KASUTATAV URIINI ÄRAVOOLUKOTT
STERIILI JA KERTAKÄYTTÖINEN VIRTSANKERÄYSPUSSI
STERILNA VREĆICA ZA DRENAŽU URINA ZA JEDNOKRATNU UPOTREBU
STERIL VIZELETGYÚJTÓ TASAK EGYSZERI HASZNÁLATRA
STERIELE URINEOPVANGZAK VOOR EENMALIG GEBRUIK
STERYLNY WOREK NA MOCZ JEDNORAZOWEGO UŻYTKU
PUNĞĂ STERILĂ PENTRU DRENAREA URINEI, DE UNICĂ FOLOSINȚĂ
STERIL URINDRÄNERINGSPÅSE FÖR ENGÅNGSBRUK

Gima 28685

Jiangsu Suyun Medical Materials Co., Ltd.
 No.18 Jin Qiao Road Dapu Industrial Park 222002 Lianyungang
 JIANGSU PROVINCE, PEOPLE'S REPUBLIC OF CHINA
 Made in China

REF

7141.11

EC REP

Shanghai International Holding Corp. GmbH (Europe)
 Eiffestraße, 80, 20537, Hamburg, Germany
 Tel. 0049-40-2513175

Gima S.p.A.
 Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

STERILE EO

0123

**Caution:**

- A. The product should be used by trained person.
- B. If you find any possible quality doubt during use, please stop using it immediately and notify our company quickly.
- C. Sterilized by EO gas. Product is sterile.
- D. For single use. Discard after use.
- E. Please checking the individual package before using it.
- F. Don't use if package is opened or damaged.
- G. Sterile and validity for 5 year. Don't use if overdue.
- H. Discard after use. After use, the products shall be collected, transported, stored and disposed to ensure that the products are harmless to human health and the environment. Or the product can be disposed in accordance with the local medical regulations for waste disposal.

Patient group: The Sterile Urine Drainage Bag is suitable for patients with uracratia, uroschisis / bladder outlet obstruction (BOO), precise monitoring of urine volume, bedridden and other lower limbs paralyzed.

Intended users: It can be used by trained doctors, nursing staff and person.

Clinical benefits: Sterile Urine Drainage Bag can be used to collect urine of patients.

Any residual risk and any undesirable side-effects: none

Install information: none

Performance characteristics: The product is provided in a sterile state and sterilized with ethylene oxide. Single-use.

Physical performance:

- a. Leak: Freedom from leakage without load/under load/under impact, the Sterile Urine Drainage Bag shall not leak.
- b. Strength: The inlet tubing shall not become detached from the bag when with a means of attaching it to the inlet tubing.
- c. Rated volume: the Sterile Urine Drainage Bag shall have a rated volume not larger than the test volume. Chemical performance: The residue of ethylene oxide in each product shall be no more than 10ug/g.

Biological performance: The product should be sterile.

Structural composition: The product is composed of Urine collection bag, inlet tubing, outlet tubing, connector, cap, leg strap and waist strap, non-return valve, attachment, ect.

Method:

- a. Before use, check whether the connection of each part of the product is perfect and whether it can operate normally.
- b. Open the small package and take the Urine Drainage Bag out.
- c. According to the clinical routine operation, connect the connector of the Urine Drainage Bag with the connector of catheter.

Storage: The product should be stored in a well-ventilated room temperature.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

ENGLISH

Product Name: Sterile Urine Drainage Bag for single use

Type and size: Pediatric 100ml general, Pediatric 100ml boy use

Indications: It is used for uracratia, uroschisis / bladder outlet obstruction (BOO), precise monitoring of urine volume, bedridden patients and other lower limbs paralyzed patients.

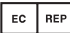
Intend purpose: The main clinical usage is to collect urine for those incontinence patients, bedridden patients and other lower limbs paralyzed patients.

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