

ENGLISH

URINE BAGS

INTENDED USE

- Use only for urine collection and draining.
- Do not use as a measuring instrument or for laboratory tests.
- Recommendations regarding volume are approximate.











DIRECTIONS FOR USE AND CONSERVATION

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

- Before use check the product integrity and its expiry date (Gima refuse all responsibility if the product is badly preserved and used inappropriately).
- The bag must always be supported, positioning it below the level of the bladder, in order to avoid reflux, in a vertical position and with the adduction tube facing upwards.
- The model of leg bag has to be fixed to the leg by elastic bands, positioning it below the level of the bladder, in order to avoid reflux.
- During use, avoid squashing, pinching or crushing the tube in order not to interrupt the flow of urine into the bag.
- Please avoid contact with chemical solvent and exposure to heat sources during storage of bags.
- Do not use with sharp tools.
- Before use discharge air if you note eventually presence of air in the bag.
- To ensure use of the entire volume of the bag, avoid the formation of air pockets even when in use.
- Before use verify that drain valve is well closed.
- Reuse of this bag could present a risk of infection.
- The bag must be replaced before the maximum level is reached.
- Firmly attach the connector to the probe or to the adaptor for the penis.

DISPOSAL INSTRUCTIONS


- Use protective gloves.
- Dispose after the first use under provisions of the law in force.

	Manufacturer	REF	Product code
	Date of manufacture	LOT	Lot number
	Disposable device, do not re-use		Expiration date
	Distributed by		Consult instructions for use
	Keep away from sunlight		Keep in a cool, dry place
	Non-sterile	MD	Medical Device
	Medical Device compliant with Regulation (EU) 2017/745		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

REF	
800160	(GIMA 28688)
800161	(GIMA 28690)
800162	(GIMA 28689)
800163	(GIMA 28692)
802220	(GIMA 28694)
802221	(GIMA 28691)
802222	(GIMA 28695)
802223	(GIMA 28693)
804257	(GIMA 28707)
804258	(GIMA 28708)

 **TECNOCLINIC, S.A.**
 Ctra. de Prats de Lluçanès, Km. 2.300 P.I.
 La Llanera 08208 Sabadell (Barcelona) - Spain
 Made in Spain

 **Gima S.p.A.**
 Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

