
ENGLISH**HYDROCOLLOID STRIPS INSTRUCTIONS FOR USE**

Hydrocolloid strips provide secure and gentle fixation for external catheters.

Please follow the securing instruction before using the product.

Hydrocolloid strips are accessories MEC and compliant with Regulation (EU) 2017/745

Soft, easy to apply, secure the MEC, Non-sterile, non-sterilizable product.

How To Apply:

1. Clean the Area before applying the Hydrocolloid strip.
2. The hair around the penis shaft may be trimmed before application.
3. Peel off one of the two paper cover strips.
4. Wind the sticky side of the strip around the penis shaft in a spiral motion starting from the back of the base and working upwards.
5. Do not make the Hydrocolloid strip too tight to prevent circulation problems.
6. The strip must overlap.
7. Before applying the MEC remove the other paper protection strip.
8. Apply the MEC leaving about 2cm between the top of the glans and the end of the MEC.

Warnings

- It is recommended that the MEC and the strip will be replaced every 12-24 hours.
- Do not over-tighten the double-sided adhesive strip to avoid circulatory problems.
- Non-sterile product, not sterilizable.
- Disposable product, do not re-use, do not recycle. After use, all medical devices that have been in contact with a patient can contain contaminants, including pathogenic microorganisms, that are difficult to eliminate.
- Store in a cool and dry place, away from sources of heat.
- After use, do not dispose to the environment.

Composition: Hydrocolloid material

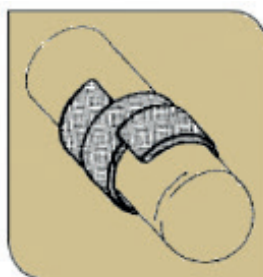
All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

Índice dei simboli - Symbol index - Index des symboles - Índice de símbolos - índice de símbolo - Symbolindex - Ευρετήριο συμβόλων - Indeks symboli - Index av symboler - Index de simboluri

	IT - Data di fabbricazione GB - Date of manufacture FR - Date de fabrication ES - Fecha de fabricación PT - Data de fabrico DE - Herstellungsdatum GR - Ημερομηνία παραγωγής PL - Data produkcji SE - Tillverkningsdatum RO - Data fabricației
	IT - Fabricante GB - Manufacturer FR - Fabricant ES - Fabricante PT - Fabricante DE - Hersteller GR - Παραγωγός PL - Producent SE - Tillverkare RO - Producător
	IT - Conservare al riparo dalla luce solare GB - Keep away from sunlight FR - À conserver à l'abri de la lumière du soleil ES - Conservar al amparo de la luz solar PT - Guardar ao abrigo da luz solar DE - Vor Sonneneinstrahlung geschützt lagern GR - Κρατήστε το μακριά από ηλιακή ακτινοβολία PL - Przechowywać z dala od światła słonecznego SE - Skyddas från solljus RO - A se păstra ferit de razele soarelui
	IT - Conservare in luogo fresco ed asciutto GB - Keep in a cool, dry place FR - À conserver dans un endroit frais et sec ES - Conservar en un lugar fresco y seco PT - Armazenar em local fresco e seco DE - An einem kühlen und trockenen Ort lagern GR - Διατηρείται σε όρσορό και στεγνό περιβάλλον PL - Przechowywać w suchym miejscu SE - Förvara på svalt och torrt ställe RO - A se păstra într-un loc răcoros și uscat
	IT - Importato da GB - Imported by FR - Importé par ES - Importado por PT - Importado por DE - Eingeführt von GR - Εισαγωγή από PL - Importowane przez SE - Importerad av RO - Importat de
	IT - Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso GB - Caution: read instructions (warnings) carefully FR - Attention: lisez attentivement les instructions (avertissements) ES - Precaución: lea las instrucciones (advertencias) cuidadosamente PT - Cuidado: leia as instruções (avisos) cuidadosamente DE - Achtung: Anweisungen (Warnungen) sorgfältig lesen GR - Προσοχή: διαβάστε προσεκτικά τις οδηγίες (επιστάσεις) PL - Ostrzeżenie - Zobacz instrukcję obsługi SE - Varsamhet: läs avsningsarna (varningar) noga RO - Atenție: Cititi și respectați cu atenție instrucțiunile (avertismentele) de utilizare
	IT - Leggere le istruzioni per l'uso GB - Consult instructions for use FR - Consulter les instructions d'utilisation ES - Consultar las instrucciones de uso PT - Consulte as instruções de uso DE - Gebrauchsanweisung beachten GR - Διαβάστε προσεκτικά τις οδηγίες χρήσης PL - Przeczytaj instrukcję użytkowania SE - Läs bruksanvisningen RO - Cititi instrucțiunile de utilizare
	IT - Non sterile GB - Non-sterile FR - Pas stérile ES - No estéril PT - Não estéril DE - nicht steril GR - όχι αποστειρωμένο PL - Nie sterylne SE - Ej steril RO - Nesteril
	IT - Numero di lotto GB - Lot number FR - Numéro de lot ES - Número de lote PT - Número de lote DE - Chargennummer GR - Αριθμός παρτίδας PL - Kod partii SE - Satsnummer RO - Număr de lot
	IT - Codice prodotto GB - Product code FR - Code produit ES - Código producto PT - Código produto DE - Erzeugniscode GR - Κωδικός προϊόντος PL - Numer katalogowy SE - Produktkod RO - Cod produs
	IT - Rappresentante autorizzato nella Comunità europea GB - Authorized representative in the European community FR - Représentant autorisé dans la Communauté européenne ES - Representante autorizado en la Comunidad Europea PT - Representante autorizado na União Europeia DE - Autorisierter Vertreter in der EG GR - Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Ένωση PL - Upoważniony przedstawiciel we Wspólnocie Europejskiej SE - Auktoriserad representant i Europeiska gemenskapen RO - Reprezentant autorizat pe teritoriul Comunității Europene
	IT - Dispositivo medico GB - Medical Device FR - Dispositif médical ES - Producto sanitario PT - Dispositivo médico DE - Medizinprodukt GR - Ιατροτεχνολογικό προϊόν PL - Wyrób medyczny SE - Medicinteknisk produkt RO - Dispozitiv medical
	IT - Dispositivo monouso, non riutilizzare GB - Disposable device, do not re-use FR - Dispositif pour usage unique, ne pas réutiliser ES - Dispositivo monouso, no reutilizable PT - Dispositivo descartável, não reutilizar DE - Für einmaligen Gebrauch, nicht wiederverwenden GR - Προϊόν μιας χρήσεως. Μην το χρησιμοποιείται εκ νέου PL - Jedno urządzenie, nie używaj ponownie SE - Engångsanordning, får ej återanvändas RO - Dispozitiv de unică folosință, a nu se refolosi
	IT - Data di scadenza GB - Expiration date FR - Date d'échéance ES - Fecha de caducidad PT - Data de validade DE - Ablaufdatum GR - Ημερομηνία λήξεως PL - Data ważności SE - Utgångsdatum RO - Valabil până la data de
	IT - Dispositivo medico conforme al regolamento (UE) 2017/745 GB - Medical Device compliant with Regulation (EU) 2017/745 FR - Dispositif médical conforme au règlement (UE) 2017/745 ES - Producto sanitario conforme con el reglamento (UE) 2017/745 PT - Dispositivo médico em conformidade com a regulamentação (UE) 2017/745 DE - Medizinprodukt im Sinne der Verordnung (EU) 2017/745 GR - Ιατρική συσκευή σύμφωνα με την ΚΑΝΟΝΙΣΜΟΣ (ΕΕ) 2017/745 PL - Wyrób medyczny zgodny z Rozporządzenie (UE) 2017/745 SE - Den medicintekniska produkten överensstämmer med förordning (EU) 2017/745 RO - Dispozitiv medical conform regulamentului (UE) 2017/745


REF 19931830 (Gima 28716)

 Mizra Medical Ltd
 21 Hasadna Street, Industrial Park, Holon, 5881411,
 Israel - Made in Israel

 RQMIS AREU SLU, Av Diagonal 409,
 Barcelona 08008, Spain

Gima S.p.A.
 Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
