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INSTALLATION OF THE DEVICE BEFORE USE

To install the device, proceed as follows:

- Insert the fitting into the lower end of the bag, screwing round (no tools required).
- Place the transparent tube onto the fitting making sure that it reaches the stop of the rubber-holder.
- Insert the other end of the transparent tube on the rubber-holder on the tap with the lever; the stretch of insertion of the tube on the tap must be such as to ensure the subsequent sealing of therapeutic liquids that will be used and avoid venting during manual operations.
- Hang the bag vertically in a suitable position using the handle supplied. The support used must ensure a firm anchoring in such a position as to avoid any contamination, or incidental introduction of liquid/foreign material inside the bag. The height for the positioning of the bag determines the pressure of the water inside of the cannula; consult your physician for the the height of the bag.
- Turn off the tap using the lever (the closed position is detectable with the 90° orientation of the lever relative to the flow of the liquid).
- Screw the two cannulas supplied (depending on use) to the tap. The device is now ready for use.

READ THE INSTRUCTIONS BEFORE USE

RETAIN THE INSTRUCTIONS, PACKAGING AND LABELING

DEVICE EVALUATED IN "CLASS I"

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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