GIMA

PROFESSIONAL MEDICAL PRODUCTS

MANIGLIONE A VENTOSA
SUCTION GRAB BAR
BARRE D'APPUI À VENTOUSE
SAUGNAPFHALTEGRIF
TIRADOR DE VENTOSA
MANILHA A VENTOSA
ΛΑΒΗ ΜΕ ΒΕΝΤΟΥΖΑ
مقبض محجمي أبير

Manuale d'uso - User manual - Notice d'utilisation - Gebrauchsanweisung
Manual de uso - Guia para utilização - Οδηγίες χρήσης - دليل الإستعمال والرعاية

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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
- The suction cup grab bar is not designed to hold or support your full body weight.
- This grab bar is designed to assist your natural motion, as when stepping into or out of a shower. Do not stress by pulling or with other movements.
- Always check the attachment and holding condition of suction cup through windows every time before each use.
- Do not attach to moving surfaces.
- Maximum User Weight : 60 kg (vertical); 45 kg (lateral)
- Only for product code 28974 - Maximum User Weight: 85 kg (vertical); 60 kg (lateral)

The manufacturer will not be held responsible for damages to people or things if the product is used for non compliant use or for incorrect or unsafe use.
 It is designed for use only on smooth and non-porous surfaces; tile, porcelain or fiberglass. Not for use on wall board, wood, painted or papered surfaces.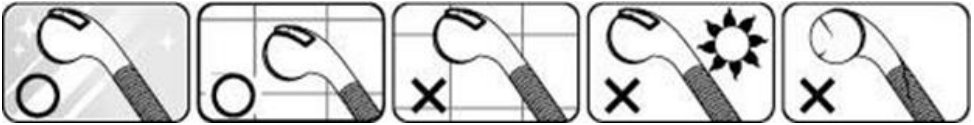

FITTING

1. Thoroughly clean and the surface with alcohol wipe and allow to dry and check the condition of the suction cups. Do not use if damaged, cracked or split.
2. Raise both levers, place the suction grab bar so that both suction cups lie firmly and completely on a smooth surface.
3. Hold one end and press on one lever with your palm, You should feel resistance as the suction cup tightens.


REMOVAL


1. Lift each of the two levers.
2. If suction remains, gently lift one edge with a finger.
3. Do not use a sharp instrument which may damage the suction cup.
4. Always to keep the levers flip up until next use.

PATENTED SUCTION INDICATOR

“GREEN” means the correct attaching, otherwise, it will turn into the “RED” request to check the mounting surface and re-attach again. Do not place over joints or grout lines.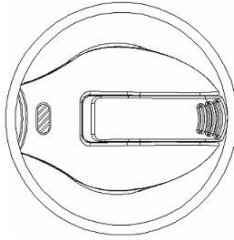

“GREEN” = OK

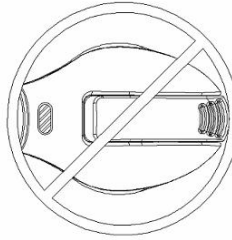“RED” = REMOUNT

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

REF 28972

REF 28973

REF 28974

Simboli - Symbols - Symboles - Symbole - Simbolos - Símbolos - Συμβολα - حرف	
	<p>IT Codice prodotto GB Product code FR Code produit DE Erzeugniscode ES Código producto PT Código produto GR Κωδικός προϊόντος</p> <p style="text-align: right;">SA كود المنتج</p>
	<p>IT Numero di lotto GB Lot number FR Numéro de lot DE Chargennummer ES Número de lote PT Número de lote GR Αριθμός παρτίδας</p> <p style="text-align: right;">SA رقم الدفعة</p>
	<p>IT Conservare in luogo fresco ed asciutto GB Keep in a cool, dry place FR À conserver dans un endroit frais et sec ES Conservar en un lugar fresco y seco DE An einem kühlen und trockenen Ort lagern PT Armazenar em local fresco e seco GR Διατηρείται σε δροσερό και στεγνό περιβάλλον</p> <p style="text-align: right;">SA يحفظ في مكان بارد وجاف</p>
	<p>IT Conservare al riparo dalla luce solare GB Keep away from sunlight FR À conserver à l'abri de la lumière du soleil DE Vor Sonneneinstrahlung geschützt lagern ES Conservar al amparo de la luz solar PT Guardar ao abrigo da luz solar GR Κρατήστε το μακριά από ηλιακή ακτινοβολία</p> <p style="text-align: right;">SA يحفظ بعيدا عن أشعة الشمس</p>
	<p>IT Leggere le istruzioni per l'uso GB Consult instructions for use FR Consulter les instructions d'utilisation DE Gebrauchsanweisung beachten ES Consultar las instrucciones de uso PT Consulte as instruções de uso GR Διαβάστε προσεκτικά τις οδηγίες χρήσης</p> <p style="text-align: right;">SA اقرأ بدقة وحرص تعليمات الاستخدام</p>
	<p>IT Fabricante GB Manufacturer FR Fabricant DE Hersteller ES Fabricante PT Fabricante GR Παραγωγός</p> <p style="text-align: right;">SA الشركة المصنعة</p>
	<p>IT Dispositivo medico conforme al regolamento (UE) 2017/745 GB Medical Device compliant with Regulation (EU) 2017/745 FR Dispositif médical conforme au règlement (UE) 2017/745 DE Medizinprodukt im Sinne der Verordnung (EU) 2017/745 ES Producto sanitario conforme con el reglamento (UE) 2017/745 PT Dispositivo médico em conformidade com a regulamento (UE) 2017/745 GR Ιατρική συσκευή σύμφωνα με την ΚΑΝΟΝΙΣΜΟΣ (ΕΕ) 2017/745</p> <p style="text-align: right;">SA جهاز طبي يتوافق مع التوجيه (UE) 2017/745</p>
	<p>IT Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso GB Caution: read instructions (warnings) carefully FR Attention: lisez attentivement les instructions (avertissements) DE Achtung: Anweisungen (Warnungen) sorgfältig lesen ES Precaución: lea las instrucciones (advertencias) cuidadosamente PT Cuidado: leia as instruções (avisos) cuidadosamente GR Προσοχή: διαβάστε προσεκτικά τις οδηγίες (ενστάσεις)</p> <p style="text-align: right;">SA الحذر: قراءة التعليمات (التحذيرات) بعناية</p>
	<p>IT Dispositivo medico GB Medical Device FR Dispositif médical DE Medizinprodukt ES Producto sanitario PT Dispositivo médico GR Ιατροτεχνολογικό προϊόν</p> <p style="text-align: right;">SA جهاز طبي</p>