GB - GIMA BRUSH: Brush for taking cytologic endocervical samples

The brush consists of nylon bristels held by a metal winding. It is fixed on a plastic stem. It is indicated for taking cytologic endocervical samples.

The device must exclusively be used by qualified, trained personnel authorised to perform the PAP test.

Do not expose to sources of heat, extract the device from the package immediately before use. Before use, visually examine the device to check its integrity

INTENDED USE: GIMA BRUSH must be used for taking uterine cervix endocervical cells

INDICATIONS: it is indicated also when there is cervical canal stenosis

CONTRA-INDICATIONS: it is not suitable during pregnancy

DIRECTIONS FOR USE: insert the speculum into the vagina to put in evidence the uterin cervix, open the package, insert the GIMA BRUSH slowly into the endocervical canal, make a complete rotation, withdraw slowly, rub the removed material on the slide.



Biogyn S.r.l. Via volta, 14 – 41037 Mirandola Italy

Distributor: Gima S.p.a. Via Marconi, 1 – 20060 Gessate Italy





MP 2005 002 rev08 del 07/21