PROFESSIONAL MEDICAL PRODUCTS

TAMPONE PER TRASPORTO MEDICALE
MEDICAL TRANSPORT SWAB
ÉCOUVILLON MÉDICAL DE TRANSPORT
HISOPO DE TRANSPORTE MÉDICO
ZARAGATOA DE TRANSPORTE MÉDICO
TUPFER FÜR DEN MEDIZINISCHEN TRANSPORT
ΙΑΤΡΙΚΟΣ ΣΤΥΛΕΟΣ ΜΕΤΑΦΟΡΑΣ ΔΕΙΓΜΑΤΟΣ
WYMAZÓWKA DO TRANSPORTU MEDYCZNEGO
ZDRAVOTNICKÝ TRANSPORTNÍ TAMPÓN
MEDICINTEKNISKA TRANSPORTPINNPROV
LÄÄKETIETEELLINEN NÄYTTEENOTTOTIKKU
MEDICINSKI TRANSPORTNI BRIS
LEKÁRSKY TRANSPORTNÝ TAMPÓN
TAMPON TRANSPORT MEDICAL
MEDISCH TRANSPORT WATTENSTAAFJE
BRIS ZA MEDICINSKI TRANSPORT
ORVOSI MINTAVEVŐ PÁLCA
MEDICINSK TRANSPORTSWAP
MEDISINSK PRØVETAKINGSPENSEL
ΤΑΜΠΟΝ ΖΑ ΜΕΔΙCΙΝΗCΚΟ ΤΡΑΝCΠΟΡΤΙΡΑΗΕ
MEDICININIS TRANSPORTAVIMO TAMPONAS
MEDICĪNISKĀIS TRANSPORTA TAMPONS
MEDITSIINILINE TRANSPORTSÖÖTMEGA TAMPOON

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REF 425102 (Gima 29753)

Jiangsu Suyun Medical Materials Co., Ltd. No.18
Jin Qiao Road Dapu Industrial Park 222002 Lianyungang
JIANGSU PROVINCE, PEOPLE'S REPUBLIC OF CHINA
Made in China

MD

EC **REP**

Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße, 80, 20537, Hamburg, Germany

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

STERILE EO

0123

Product Name:

Medical Transport swab

Type:

I-1,I-2,I-3,I-4,I-5,I-6, II-1,II-2,II-3,II-4,II-5,II-6, II-7,II-8,II-9,II-10,II-11,II-12.

Indications:

This product is suitable for gynecological disease and ENT lesions sampling.

Intend purpose:

Transport Swab intended to use for gynecological disease or ENT lesions sampling.

Contraindications:

None.

Caution:

- A. Only for single use. Use as soon as pouch opened. Discard after use.
- B. Do not use if previously opened or if pouch damaged.
- C. Guarantee contents non-contaminated and package easy to open before expiry date.
- D. Protect product against being humidified, heated and contaminated during storage and transport.

Patient group:

It is suitable for people who require gynaecology and ENT examination.

Intended users:

It can be used by trained doctors, nursing staff.

Clinical benefits:

The product plays an auxiliary role in the treatment of diseases.

Any residual risk and any undesirable side-effects:

none

Install information:

none

Performance characteristics:

The product is provided in a sterile state and sterilized with ethylene oxide. Single-use

Physical performance:

- A. The plastic part is complete, without obvious defects such as sharp edges and burrs, and there is no obvious bending phenomenon in the plastic part.
- B. The surface is clean and free from obvious foreign objects and impurities.

Structural composition:

The product are composed of cotton tip with plastic stick.

Method:

- A. The product is intended to use for gynecological disease and ENT lesions sampling.
- B. The cotton swab is used for sampling and the outer tube used for loading samples, the medium works as Nutrient Fluid.
- C. Send the samples to the pathology room for analysis after sampling.

Storage:

The product should be stored in a well- ventilated room temperature.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

Indice dei simboli - Symbol index - Index des symboles - Índice de símbolos - Índice de símbolo - Symbolindex - Ευρετήριο συμβόλων - Indeks symboli - Index symbolů - Symbol index - Symboli-indeksi - Indeks simbolov - Index symbolov - Index de simbol - Symbool index - Indeks simbola - Szimbólum index - Symbolindeks - Indeks over symboler - Индекс на символа - Simbolių rodyklė - Simbolu rādītājs - Sūmbolīte indekss

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