STERILE GYNAECOLOGICAL COLLECTORS FOR SINGLE USE
RACCOGLITORE GINECOLOGICO STERILE PER USO SINGOLO
СТЕРИЛНИ ГИНЕКОЛОГИЧНИ КОЛЕКТОРИ ЗА ЕДНОКРАТНА УПОТРЕБА
STERILNÍ GYNEKOLOGICKÉ KOLEKTORY NA JEDNO POUŽITÍ
STERILE GYNÆKOLOGISKE SAMLERE TIL ENGANGSBRUG
STERILE GYNÄKOLOGISCHE SAMMELBEHÄLTER FÜR DEN EINMALIGEN GEBRAUCH
ΑΠΟΣΤΕΙΡΩΜΕΝΟΙ ΓΥΝΑΙΚΟΛΟΓΙΚΟΙ ΣΥΛΛΕΚΤΕΣ ΜΙΑΣ ΧΡΗΣΗΣ
COLECTORES GINECOLÓGICOS ESTÉRILES DE UN SOLO USO
STERIILSED GÜNEKOLOOGILISED KOGUJAD ÜHEKORDSEKS KASUTAMISEKS
STERIILIT GYNEKOLOGISET KERÄILIJÄT KERTAKÄYTTÖÖN
COLLECTEURS GYNÉCOLOGIQUES STÉRILES À USAGE UNIQUE
STERILNI GINEKOLOŠKI KOLEKTORI ZA JEDNOKRATNU UPOTREBU
STERIL NŐGYÓGYÁSZATI KOLLEKTOROK EGSZERI HASZNÁLATRA
STERILŪS VIENKARTINIAI GINEKOLOGINIAI KOLEKTORIAI
STERILI GINEKOLOGISKIE KOLEKTORI VIENREIZĒJAI LIETOŠANAI
STERILE GYNEKOLOGISKE SAMLERE FOR ENGANGSBRUK
STERIELE GYNAECOLOGISCHE COLLECTOREN VOOR EENMALIG GEBRUIK
STERYLNE KOLEKTORY GINEKOLOGICZNE JEDNORAZOWEGO UŻYTKU
COLETORES GINECOLÓGICOS ESTÉREIS PARA USO ÚNICO
COLECTOARE GINECOLOGICE STERILE DE UNICĂ FOLOSINȚĂ
STERILNÉ GYNEKOLOGICKÉ KOLEKTORY NA JEDNO POUŽITIE
STERILNI GINEKOLOŠKI ZBIRALNIKI ZA ENKRATNO UPORABO
STERILA GYNEKOLOGISKA UPPSAMLARE FÖR ENGÅNGSBRUK

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Jiangsu Suyun Medical Materials Co., Ltd.
No.18 Jin Qiao Road Dapu Industrial Park 222002 Lianyungang
JIANGSU PROVINCE, PEOPLE'S REPUBLIC OF CHINA
Made in China

Gima 29826 - 29827

4204 11 - 4204 14

Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße, 80, 20537, Hamburg, Germany
Tel.0049-40-2513175

0123

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

IIIB- 3, IIIB- 4, IIIB- M1, IIIB- M2, IIIB- M3, IIIC, IIID, IIID- 2

Metodo:

- a. Use the dilator to open the vagina at an appropriate angle.
- b. Determine the location to be inspected or sampled.
- c. Deepen the sampling head of the gynecological sampler to the sampling site and take a rotating or wiping flat method to sample.

Contraindications:

No special contraindication.

Caution:

- a. When the product is used, it should strictly implement the requirements of aseptic operation specifications and related regulations, and be used by trained doctors and nurses. In the course of use, the operator or user's body should be treated in a timely manner according to medical regulations.
- b. If you find any possible quality doubt during use, please stop using it immediately and notify our company quickly.
- c. This product is used for single use and is destroyed after using; if the small package is damaged and the product is prohibited to be used; this product is sterilized by ethylene oxide and the product is sterile; the product is valid for 5 years and should not be used after it expires.
- d. Appropriate types and specifications should be selected according to the patient's body type.
- e. If there are burrs, cracks, or nodes in the application area of the gynaecological collector, they should not be used.
- f. This product shall be disposed of according to the requirements of waste disposal in the local medical regulations after using.

Storage:

The product should be stored in a room with no corrosive gas and well ventilated, with adequate protection for the product.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

ENGLISH**Intended purpose:**

The main clinical usage of Sterile gynaecological collectors for single use is to sample for gynecology and venereal disease.

Indications:

Women who need gynecological examination, such as vaginal examination and cervical examination.

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