

(GB)

CANNULA RIGIDA CURVA

Straight/curved rigid cannula for intrauterine aspiration and for voluntary abortion.

The device must exclusively be used by specialised medical personnel

Disposable device: any reuse could cause the risk of cross-contaminations and/or failed sterility or cleaning.

• Do not expose to sources of heat; • Extract the device from the package immediately before use; • Before use, visually examine the device to check its integrity


Intended use: it is used for for endouterin suction in the voluntary abortion.

Indications: it is indicated for endouterin suction in the voluntary abortion

Contraindications: it is contraindicated in case of severe vaginal, cervical and/or pelvic infection.

Directions for use: ▪ After anaesthetising the patient, ▪ Dilate the cervical canal, ▪ Select a cannula of suitable diameter, ▪ Extract the cannula from the package, ▪ Fix the cannula to the handpiece, ▪ Insert the cannula into the uterine cavity, ▪ Proceed with aspiration.

(I)	Caratteristiche che deve avere il manipolo della pompa aspirante per ottenere una combinazione sicura tra il raccordo della cannula e il manipolo della pompa aspirante:
(GB)	Characteristics the suction pump handpiece must have to obtain a secure fitting between the cannula connector and the suction pump handpiece:
(F)	Caractéristiques que la pièce à main de la pompe aspirante doit présenter pour obtenir un ensemble fiable entre le raccord de la canule et la pièce à main.
(D)	Erforderliche Merkmale des Handstücks der Absaugpumpe für eine sichere Verbindung zwischen dem Anschluß der Kanüle und Handstück der Absaugpumpe.
(E)	Características que debe presentar la pieza manual de la bomba aspirante para obtener una una combinación segura entre la misma y la conexión de la cánula
(P)	Características que deve ter o manípulo da bomba de aspiração para haver combinação segura entre o conector da cânula e o manípulo da bomba de aspiração
(CZ)	Charakteristiky, které musí splňovat násadec aspirační pumpy, pro zaručení bezpečného spojení spojky kanyly a násadce aspirační pumpy

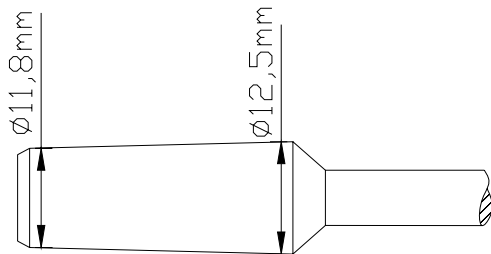

CANNULA RIGIDA CURVA

REF	Ø
CRC 07 S01	7 mm
CRC 08 S01	8 mm
CRC 09 S01	9 mm
CRC 10 S01	10 mm
CRC 11 S01	11 mm
CRC 12 S01	12 mm



Biogyn s.n.c

Via A. Volta, 14 41037
Mirandola (MO) - Italy



Distributore:

GIMA S.P.A.

Via Marconi, 1

I-20060 Gessate (MI) – Italy

www.gimaitaly.com

