

REF. 0111 e 0111/L

HOSPITAL'S CLAMP

INSTRUCTIONS FOR USE

WARNINGS

- The device is supplied sterile.
- Do not use if the package is not intact.
- Use immediately after opening the enclosure.
- The device should only be used by qualified medical personnel.
- To be used only for the umbilical cord.
- The product is single use. Do not use after use.

INDICATIONS OF USE

The device is intended for use in obstetrics the closure of the umbilical cord.

METHOD OF USE

Remove the product from the sterile envelope.

Place two umbilical tweezers on the cord, one about ten centimeters from the baby's body, the other at about fifteen centimeters (the distance between tweezers must be about five centimeters).

Take the two ends of the tweezers with two fingers and after placing it on the umbilical cord, tighten until it clicks into place.

MAKE SURE THAT THE TWEEZER IS TIGHTENED AND THE STOPS ARE WELL ENTANGLED BETWEEN THEM.

Cut the umbilical cord still attached to the placenta (the placenta is naturally expelled at most twenty minutes after the baby's exit).

The tweezers can only be removed by pulling the back of the clip.

WARNINGS

- Periodically check the tightness of the tweezers on the newborn at short intervals of time until the cord has healed.
- In the event that continuous control is impossible, it is advisable to apply an additional safety forceps.

STORAGE / STORAGE

Store in a dry place. Do not expose to temperatures above 50 ° C.

ELIMINATION AND DISPOSAL AFTER USE

The product is single use so it is not reusable, it can be discharged or burned together with other waste, observing the provisions of the Local Authorities. Elimination by incineration does not give rise to the formation of toxic residues.

 **AK Medical S.r.l. Unipersonale**

Via del Chioso 10 - 24030 Mozzo (BG) - Italy

Tel: 035 613712 - Fax: 0544 1761249

Cell: 334 1807240 - www.akmedical.it

info@akmedical.it

MADE IN CHINA

