



Via Del Chioso n.8
24030 MOZZO (BG) - ITALY
Tel.: ++39/035/4376456
Fax: ++39/035/617844
E-mail: gps@gpsmedical.com
website: www.gpsmedical.com

STERILE UMBILICAL CORD CLAMP INSTRUCTION FOR USE

Read the instructions before using the medical device.

Do not reuse. Sterility guaranteed only with undamaged package. Use immediately after opening

Check always the integrity of the locking system before use. If incomplete, do not use.

For good measure use two umbilical clamps.

Place the clamps on the umbilical cord, the first at 1e cm from the newborn baby's body, the second at 15 cm (the distance between the two clamps should be approx 5 cm).

Close the first clamp until it locks and clicks; make certain the two parts of the clamp are perfectly locked together. Do the same with the second clamp .

Cut the cord.

The clamp can be removed only breaking its back side.

Risk condition: bleeding from umbilical cord may be possible.

Dispose of after use according to the laws in force. For use only under trained medical staff.

