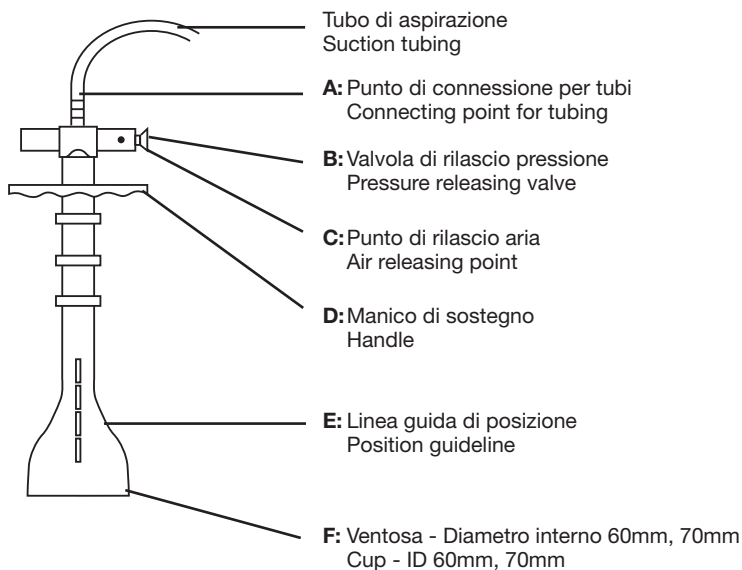
# **VENTOSA FETALE VACUUM**

## **FETAL VACUUM CUP**

### **Manuale d'uso - User manual**

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# DESCRIPTION

Fetal Vacuum Cup is for birth assistance. Available for two sizes, 60mm, and 70mm ID. Made of medical grade silicone rubber.  
Soft suction cup allow for easy insertion, and blue guideline is designed for positioning reference. Polysulfone handle for light weight and autoclavable.

# OPERATION INSTRUCTIONS

1. Connect the tubing to both the suction point of the suction machine and connecting point of the suction cup (A).
2. Turn on the suction pump.
3. Attach the cup (F) to the fetal head. The fetal head will be gripped once been attached.
4. Press pressure releasing valve (B) to release fetal head.

# CAUTIONS

1. U.S. Federal law restricts this device to sale by or on the order of a physician (USA only).
2. Don't apply this device for a long time after gripping the target. It may cause bruise of fetal.
3. Don't apply it on vulnerable area, such as eyes, mouths, and temples. Only apply it on area with bones.
4. After gripping the fetal head, make sure the guideline is straight when pulling it . Don't twist the cup.
5. Steam Sterilization is typically carried out in an autoclave at 121°C (250°F) for 15-75 minutes.

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29980 / VC-95200  
29981 / VC-95300

Besmed Health Business Corp.  
No. 5, Lane 116, Wu-Kong 2nd Road, Wu-Ku District,  
New Taipei City 24888, Taiwan  
Made in Taiwan

Mdi Europa GmbH - Langenhagener Str. 71  
30855 Hannover-Langenhagen - Germany

CE 2460