

Description/Intended Use

These devices are single-use supplied sterile.

Contraindications

These devices should never be used when:

- There is visible evidence of damage to the exterior of the device such as cracked plastic or connector damage.
- These devices fail the inspection described herein.
- In the presence of flammable gases, liquids, and/or oxygen enriched environments.

Safety Tips

- Use lowest possible power setting on electrosurgical unit capable of achieving desired surgical effect.
- Never allow the cables connected to these devices to be in contact with skin of the patient or operator during electrosurgical activations.
- Do not permit the cables connected to these devices to be parallel and in close proximity to the leads of other electrical devices.
- Always place unused electrosurgical accessories in a safe insulated location such as a holster when not in use.
- Inspect and test each device before each use.
- Discard devices that have reached their life expectancy.
- Activation of an electrosurgical device when not in contact with target tissue or in position to deliver energy to target tissue (fulguration) may cause capacitive coupling.










Inspection

These devices should be inspected before and after each use. Visually examine the devices for obvious physical damage including:

- Cracked, broken or otherwise distorted plastic parts.
- Broken or significantly bent connector contacts.
- Damage including cuts, punctures, nicks, abrasion, unusual lumps, significant discoloration.
- Tips for damage, corrosion or misalignment condition

Storage & Handling


Electrodes (single use) must be stored in a clean, cool and dry area. Protect from mechanical damage & direct sun light. Handle with extreme care.

	Keep in a cool, dry place		Keep away from sunlight	STERILE EO	Sterilized using ethylene oxide
	Manufacturer		Date of manufacture		Caution: read instructions (warnings) carefully
REF	Product code	LOT	Lot number	EC REP	Authorized representative in the European community
	Consult instructions for use		Disposable device, do not re-use	CE	Medical Device complies with Directive 93/42/EEC
	Temperature limit		Expiration date		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

REF 30420 - 30422 - 30425 - 30429 - 30678

 Tecno Instruments Pvt. Ltd.
316C Small Industrial Estate
Sialkot, 51340 (PAKISTAN)

EC REP Obelis s.a.
Bd. General Wahis 53
1030 Brussels (BELGIUM)

CE 1639

Importato da / Imported by:

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

STERILE EO 

