



REF 20h
00900.FU01

REF 5h
00900.FST

www.led.it

EN 00900.FST - First ULPA Filter (5h) - BLU
00900.FU01 - ULPA Filter for SURTRON EVAC (20h) - RED

Verify the status of the product before the use.
Disposable product. No reusable.
Do not draw liquids.
Product should be disposed of in accordance with user procedures for the control of hazardous waste.

The connections must be executed by trained staff.


The product is compatible for the use with the smoke evacuators SURTRON® EVAC (10200.10 - 10200.10/115V).

For the connections please follow the instructions of the smoke evacuator units SURTRON® EVAC.

Technical Data

Filter Type: ULPA with active carbon
Filter efficiency: 99,999X %
Particles diameter: 0,12 micron
Filter diameter connection: 22 mm (7/8")
Filter Duration 00900.FST: 5 hours
Filter duration 00900.FU01: 20 hours


LED SpA
Via Selciatella 40
04011 Aprilia (LT) - ITALY

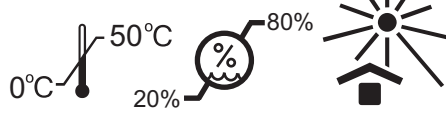

IT EN ES FR DE PT TR

IA00900-XXXAr2
ed. 07/2013 2013 (C) LED SpA

FIG.-IM.-BILD-GRAFIC A

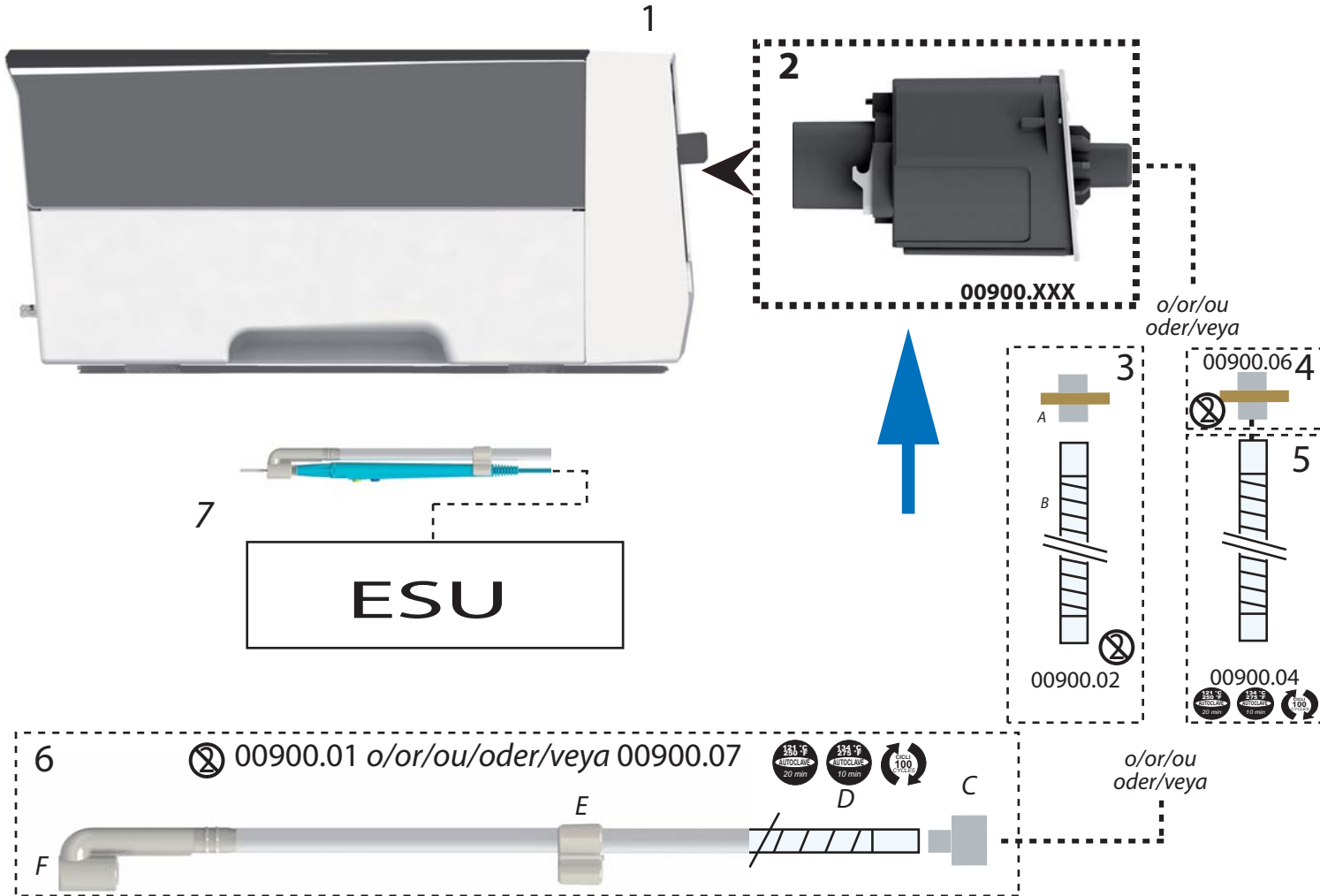

1 SMOKE EVACUATOR UNIT* serie 10200.10 - 10200.10/115V
2 FILTER series 00900.XXX
3 ASPIRATION KIT (00900.02)* - A Filter - B Pipe dia. 22mm
length 2,4 mt (100")
4 Filter dia. 22 mm (00900.06)*
5 Autoclavable Tube dia. 22mm (00900.04)*
6 HANDLE ASPIRATION KIT (00900.01 or 00900.07)* - C Adapter 10mm/22mm - D
Pipe Length 1,5 mt (59") - E Clip to fix the pipe on the handle - F Adapter for the
handle.
7 Detail of the handle* with 00900.01 or 00900.07, the handle should be connected to
your electrosurgical unit (ESU)*. *not furnished



REF 20h
00900.FU01

REF 5h
00900.FST

LED SpA
Via Selciatella 40
04011 Aprilia (LT) - ITALY



IA00900-XXXAr2
ed. 07/2013
2013 (C) LED SpA