



www.led.it

EN ASPIRATION KIT for SURGICAL SMOKE EVACUATOR

REF 00900.02



LED SpA
Via Selciatella 40
04011 Aprilia (LT) - ITALY



NON
STERILE



50°C
0°C
RH
20% - 80%



IT EN ES FR DE PT TR

IA00900-01Ar2
ed.05/2012 2012 (C) LED SpA


Verify the status of the product before the use.
Disposable product. No reusable.
Do not draw liquids .
Product should be disposed of in accordance with user procedures for the control of hazardous waste.

The connections must be executed by trained staff.

The product is furnished for the draw of the surgical smokes during the "open surgery" procedures (ex. laparotomia): The typical connection is:
-connection to the code 00900.01 (furnished separated) (fig. A -part.4)
-connection to the filter series XXX00900.XXX (furnished separated (fig. A -part.2)

For the connection to an endoscopic accessory (for example laparoscopic) or another compatible captation system for diameter of connection (22mm).

FIG.-IM.-BILD-GRAFIC A

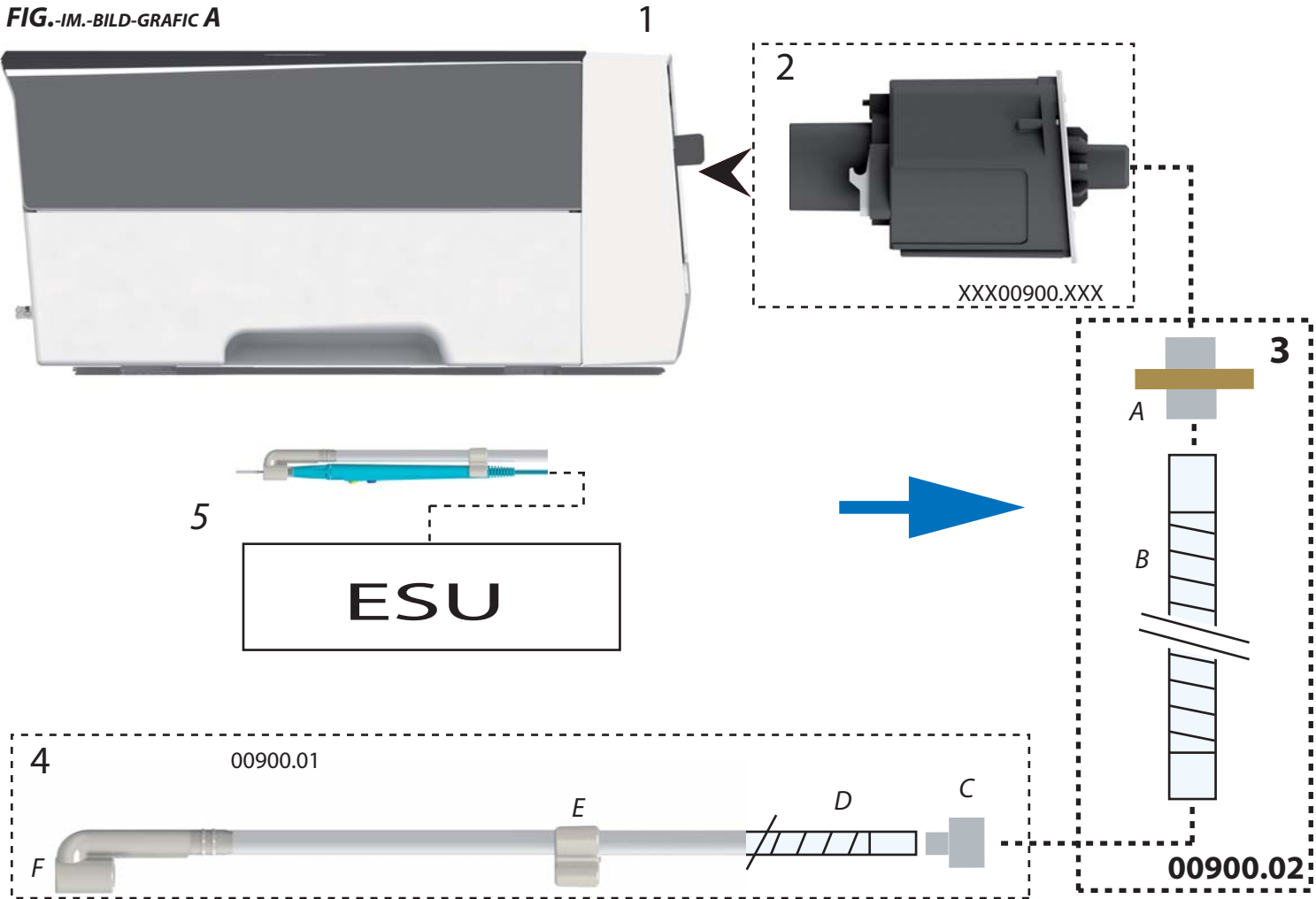

1 SMOKE EVACUATOR UNIT* serie XXX10200.10 - XXX10200.10/115V
2 FILTER* series XXX00900.XXX
3 ASPIRATION KIT (00900.02) - A Filter - B Pipe dia. 22mm
length 2,4 mt (100")
4 HANDLE ASPIRATION KIT (00900.01)* - C Adapter 10mm/22mm - D Pipe
dia. 10 Lenght 1,8 mt (72") - E Clip to fix the pipe on the handle - F
Adapter for the handle.
5 Detail of the handle* with 00900.01, the handle should be connected to
your electrosurgical unit (ESU)*. *not furnished



REF 00900.02

LED SpA
Via Selciatella 40
04011 Aprilia (LT) - ITALY



IA00900-02Ar2
ed.05/2012 2012 (C) LED SpA