

CLEANING

Cables can be cleaned with a soft cloth soaked with disinfecting solution.

LIFE OF PRODUCT

The life of the products depends on the number of uses and on maintenance type. The products must be immediately destroyed in case of deterioration of the material.

STORAGE

The product must be stored in the original package at the environmental conditions (temperature and relative humidity) specified on the pouch's label. Putting external heavy weights on the package, the product could be damaged.

WASTE DISPOSAL

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority. For any malfunction or defect of the device, inform the Manufacturer's Quality Service.

WASTE DISPOSAL

Refuses deriving from health structures must be disposed in according to the regulation in force..



DIRECTIONS FOR USE CONNECTING CABLES BETWEEN PADS AND ELECTROSURGERY DEVICES

DESCRIPTION AND INDICATIONS

The packaged product is sold in is **NOT STERILE**.

CONNECTING CABLES WITH TAB CONNECTIONS FOR DISPOSABLE PADS FOR ELECTROSURGERY. On one end the cables are fitted with a clamp connection, and on the other, with various connectors which make the cables compatible with numerous electrosurgery devices.

MODELS F7902 – F7902-S – F7904 - F7902/3MT

Connecting cable with a mono Ø 6,3 mm jack connection

MODELS F7903

Connecting cable with Valleylab-type connection

MODELS F7902/4

Connecting cable with male Ø 4,0 mm plugs

MODELS F7902/24

Connecting cable with 2 male Ø 4,0 mm plugs

MODELS F7903/F

Connecting cable with Valleylab-type connector for Force 1-2-4-4B models

CONNECTING CABLES FOR RE-USABLE PADS FOR ELECTROSURGERY WITH FEMALE 4MM CONNECTORS

On one end the cables are fitted with 4 mm safety terminals for connecting to the pads, and on the other with various connections which make the cables compatible with numerous electrosurgery devices.

MODELS F7922 – F7924

Connecting cable with a mono Ø 6,3 mm jack connection

MODELS F7923

Connecting cable with Valleylab-type connection

MODELS F7922/4

Connecting cable with male Ø 4,0 mm plugs

MODELS F7922/24

Connecting cable with 2 male Ø 4,0 mm plugs

MODELS F7923/F

Connecting cable with Valleylab-type connection for Force 1-2-4-4B models

INSTRUCTIONS FOR USE

Carefully check packaging, cables and connection plugs. Do not use the product in case of visible damages or defects.

Connecting of cables with clamp connection:

- raise the moveable lever positioned on the blue plastic part of the clamp.
- insert the tab of the disposable plate into the receptacle located on the blue plastic part of the clamp.
- replace the lever in the home position.

Connecting of cables with 4 mm diameter male plugs:

- Insert the plugs into the holes located on the pad;
- On being inserted the protective sheath on the tip of the plug retracts in order to allow for connection to the pad.

WARNING

- Connect the clamp to the tongue of the disposable plate, being careful to position the clamp outside the gel-covered area.
- **The product is supplied non-sterile.**
- The equipment and plates used with this product must conform to the regulations currently in force.

NB: The products must be used by qualified healthcare personnel.

Do not pull cables at the insulation

	IT	EN	FR	DE	ES	RU	PT	CS
	Conforme alla vigente normativa Europea sui Dispositivi Medici	Compliant with current European legislation on Medical Devices	Conforme à la législation européenne en vigueur sur les dispositifs médicaux	Entspricht der aktuellen europäischen Gesetzgebung für Medizinprodukte	Cumple con la legislación europea vigente sobre dispositivos médicos	Соответствует действующему европейскому законодательству о медицинских устройствах	Em conformidade com a legislação europeia em vigor sobre Dispositivos Médicos	Vyhovuje současně evropské legislativě o zdravotnických prostředcích
	Dispositivo medico	Medical Device	Dispositif médical	Medizinprodukt	Producto sanitario	Медицинское устройство	Dispositivo médico	Zdravotnický prostředek
	Identificativo unico del dispositivo	Unique Device Identifier	Identifiant unique des dispositifs	Einmalige Produktkennung	Identificador único del producto	Уникальный идентификатор устройства	Identificação única do dispositivo	Jedinečným identifikátorem prostředku
	Attenzione, consultare la documentazione allegata	Caution, consult accompanying documents	Attention, lire attentivement les instructions	Achtung, Gebrauchsanweisung lesen	Atención, leer atentamente las instrucciones	Внимание, обратитесь к сопроводительной документации	Atenção, ler atentamente as instruções	Varování, čtěte průvodní dokumentaci
	Consultare le istruzioni d'uso	Consult instructions for use	Instructions d'utilisation	Gebrauchsanweisung lesen	Consulte las instrucciones de uso	Обратитесь к инструкции по применению	Consulte as instruções de utilização	Varování, čtěte průvodní dokumentaci
	Fabbricante	Manufacturer	Fabricant	Hersteller	Fabricante	Производитель	Fabricante	Výrobce
	Numero di catalogo	Catalogue number	Code de référence	Katalognummer	Número de Catálogo	Каталожный номер	Número de Catálogo	Katalogové číslo
	Numero di lotto	Batch code	Numéro de lot	Posten-Nummer	Número de Lote	Код партии	Número de Lote	Číslo šarže
	Data di produzione	Date of manufacture	Date de production	Herstellungsdatum	Fecha de Producción	Дата изготовления	Data de Fabrico	Datum výroby
	Scadenza	Use by	Date de péremption	Verfalldatum	Fecha de Caducidad	Использовать до	Prazo de Validade	Spotřebujte do
	Limiti di temperatura	Temperature limitation	Limites de température	Temperaturbereich	Límites de Temperatura	Температурные ограничения	Limites de Temperatura	Omezení teploty
	Limiti di umidità	Humidity limitation	Limites d'humidité	Feuchtigkeitsbereich	Límites de Humedad	Ограничения по влажности	Limites de Humidade	Omezení vlhkosti
	Numero di pezzi	Quantity of pieces	Nombre de pièces	Stückzahl	Cantidad de piezas	Количество штук	Quantidade de peças	Počet kusů