

# FOOTSWITCH



UDI

REF 00304.04

2024



1-2 cc  
4 nc  
3-5 switch



IP 67  
WATER PROOF



cable 3 mt MADE IN ITALY

LED SpA  
Via M.T. Cicerone 138  
03100 Frosinone (LT) - ITALY



-10°C

50°C

10% 85%

IT EN ES FR DE PT PL

IA00304-04b\_ML1  
ed. 2022-10 2022 (C) LED SpA

EN WATER-PROOF FOOT SWITCH (5 pin) physio3 mt

The product is supplied for the foot control of compatible electronic units.

Verify the status of the product before the use. Throw away the defective products.

This product is compatible with versions of equipment that require it, refer to the instructions for the unit.

The connections must be executed by trained staff.

Installation:

- Insert the product connector on specific electronic unit connector, paying attention to the direction of insertion (fig. A -part. 1-2).
- to lock the product, turn the dial on the unit (fig. A - part. 3).

Turn on the unit and perform functional tests by acting on the pedal (fig. A part 4).

FIG.-IM.-BILD-GRAFIC-RYS. A



EN PRESS = ACTIVATION / RELEASE = NO ACTIVATION