

FOOTSWITCH

REF 00305.06_S

1-2 cut
2-3 coag
4-5 cc
6 nc



LED SpA
Via M.T.Cicerone 138
03100 Frosinone (LT) - ITALY

IT EN ES FR DE PT PL

SURGEON

IP 67
WATER PROOF




EXPLOSION PROOF

cable 5 mt



MADE IN ITALY

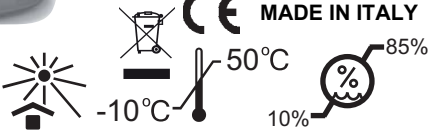

IA00305-06_Sc_ML1
ed. 2022-1& 2022 (C) LED SpA

EN DOUBLE WATER-PROOF FOOT SWITCH (TOUCH / TOUCH HP 300-400)

The product is supplied for the foot control of compatible electronic units.

Verify the status of the product before the use. Throw away the defective products.

This product is compatible with versions of equipment that require it, refer to the instructions for the unit.

The connections must be executed by trained staff.

Installation:

- Insert the product connector on specific electronic unit connector, paying attention to the direction of insertion (fig. A -part. 1-2).
- to lock the product, turn the dial on the unit (fig. A - part. 3).

Turn on the unit and perform functional tests by acting on the pedal (fig. A part 4).

FIG.-IM.-BILD-GRAFIC-RYS. A



EN PRESS = ACTIVATION / RELEASE = NO ACTIVATION
YELLOW= CUT / BLUE = COAG