

Instruction for Use Electrodes (Single Use)

Description/Intended Use:

These devices are single-use supplied sterile.

Contraindications:

These devices should never be used when:

- There is visible evidence of damage to the exterior of the device such as cracked plastic or connector damage.
- These devices fail the inspection described herein.
- In the presence of flammable gases, liquids, and/or oxygen enriched environments.

Safety Tips:

- Use lowest possible power setting on electro-surgical unit capable of achieving desired surgical effect.
- Never allow the cables connected to these devices to be in contact with skin of the patient or operator during electro-surgical activations.
- Do not permit the cables connected to these devices to be parallel and in close proximity to the leads of other electrical devices.
- Always place unused electro-surgical accessories in a safe insulated location such as a holster when not in use.
- Inspect and test each device before each use.
- Discard devices that have reached their life expectancy.
- Activation of an electro-surgical device when not in contact with target tissue or in position to deliver energy to target tissue (fulguration) may cause capacitive coupling.

Inspection:

These devices should be inspected before and after each use. Visually examine the devices for obvious physical damage including:

- Cracked, broken or otherwise distorted plastic parts.
- Broken or significantly bent connector contacts.
- Damage including cuts, punctures, nicks, abrasion, unusual lumps, significant discoloration.
- Tips for damage, corrosion or misalignment condition.

Storage & Handling:

Electrodes (single use) must be stored in a clean, cool and dry area. Protect from mechanical damage & direct sun light. Handle with extreme care.

Warranty:

These products are guaranteed against material and workmanship. The warranty is null and void should damage occur as a result of improper handling use. Care must be taken in the use and reprocessing of these products.



Consult Instructions for Use



Keep Dry



Latex Free



Temperature Limitations



Single Use



Keep away from Sunlight



STERILE - STERILIZED BY ETHYLENE OXIDE