

MAGNIFIER LED LAMP

1 Cautions:

- 1) Do not use in wet environments (like the bathroom). Clean your skin before operating.
- 2) Turn the power off after operation.
- 3) Please disconnect the product from power source if it is not being used for a long period of time. Clean the lamp head and cover it properly for storage.
- 4) Please wipe the lens with soft lens cleaning paper when cleaning the lens. Do not wipe with cloth and paper of other materials, otherwise the lens may be damaged.
- 5) Avoid water or other liquid contact with the brightness switch when adjusting the device.
- 6) Please avoid direct light projection to the eye.
- 7) If the external soft power cable is damaged, it can only be replaced by the manufacturer, service agent or personnel with similar qualification in order to avoid hazard.
- 8) WARNING "⚠" (IEC 60417-6042): The light source can only be replaced by the manufacturer, service agent or personnel with similar qualification.

2 Instruction

This device mainly detects changes in the human skin surface by examining the various minor changes in the skin surface. Providing further effectiveness when operating beauty and care treatments.

The magnifying function of the product can also be used for beauty treatments such as eyebrow and lips tattooing and eyelash grafting, etc.

The magnifying lens can be use independently with the supporting stand or in combination with other devices.

The device has an innovative design and elegant profile. It has a safety design to avoid injury by clamping during operation. The magnifying lens is made of ultra-white glass with 3 Diopter, better visual effect without phenomenon of distortion.

3 Technical Specifications

- 1) See the table below for input power:

Rated voltage (V)	100~240V
Rated frequency (Hz)	50/60Hz

- 2) Rated power and illumination: (illumination tested at a distance of 20cm)


Level 1: 2.5W, 950 LUX
Level 2: 3.5W, 1400 LUX
Level 3: 5.0W, 1900 LUX
Level 4: 7.0W, 2400 LUX

- 3) Net weight: 12.8kg
- 4) Gross weight: 15kg
- 5) Package dimensions: 83.5cm x 69.5cm x 13.5cm


4 Directions of Use:

- 1) Plug in the cold light lamp to a power source.
- 2) Turn the power switch to ON, flip the dust cover on the upper lid and aim the magnifying lens at the skin surface that needs to be examined or undergo beauty treatment.
- 3) Adjust the distance between the cold light lamp and skin (10~20cm will be a best visual effect distance) with proper brightness to achieve the clearest state. (Total 4 levels for adjustment, specific methods are as follows.)
- 4) Examine the skin or begin beauty treatment. Brightness can be adjusted during operation.

Brightness Up Brightness Down

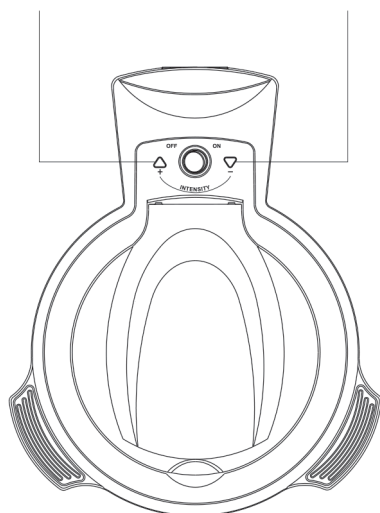


5. Magnifying Lens:

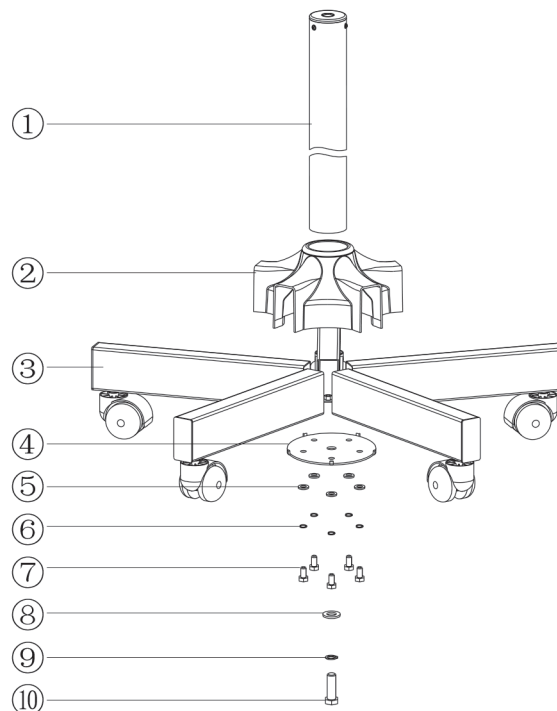

Installation Parts:

No.	Name	Qty
1	Column	1
2	Pentagonal base holder	1
3	Five-legged stand	1
4	Pentagonal clamp plate	1
5	Spring pad φ6	5

No.	Name	Qty
6	Hexagonal screw M6X20	5
7	Flat pad φ10	1
8	Spring pad φ10	1
9	Hexagonal screw M10X40	1

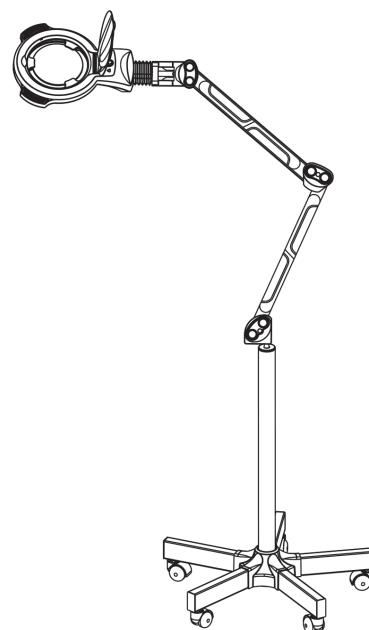

Disposal: The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

Manufacturer **SILVERFOX CORPORATION LIMITED**
No. 18, 1st TongLe Road, TangXia Town, Pengjiang District,
JiangMen City, Guangdong Province, China
amy.liang@silverfox.cn
Made in China

GIMA 30736

REF 1006A

Imported by:

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

