

LUXIFLEX LED PLUS

MIMSAL

MIMSAL TRADE S.L.

C. Mollet 17, Pol. Ind. Palou Nord - 08401 Granollers (Barcelona) - Spain
Tel. +34 930 139 860 - mimsal@mimsal.com - www.mimsal.com



ver. 1

MIMSAL

The design of the light fixture is highly versatile. It is suitable for medical examination and has many features that makes it ideal for use in medical centres and veterinary clinics.

The optimum working distance between the light bulb and the object being lit up is 50 cm.

1. Installation

- Requires a MIMSAL mount, which is installed before the lamp is placed in position.
- A table clamp mount is supplied and wall or ceiling mounts or a rotating base can be supplied as optional accessories.
- MIMSAL cannot guarantee the life and functionality of the product where third party accessories are used.

2. Caution

- Always disconnect the lamp from the power supply when cleaning or carrying out maintenance work.
- Clean either dry or using a weak solution of normal soap and water. Avoid using alcohol, acids or any type of organic solvent.
- Do not leave on for longer than 8 hours at a time.

3. Troubleshooting and maintenance

- Only for qualified electrical engineers or other qualified personnel.

Please contact your MIMSAL rep if you have any other problem.

MIMSAL only accepts responsibility for maintenance, repair or changes carried out by MIMSAL staff using original MIMSAL parts.

The manufacturer reserves the right to change the characteristics of its products without prior notice.

	LUXIFLEX LED PLUS
Light bulb	LED 10W
Useful life	> 50.000 h
Lux at 50 cm	35.000 lux
Colour temperature	4.500° K
Colour Rendering Index	97%
Electrical Protection Class	Class II
Length of Arm	72 cm
Color	White
Power	110/230V
Frequency	50/60 Hz