
# GIMA

## HEAD LOUPE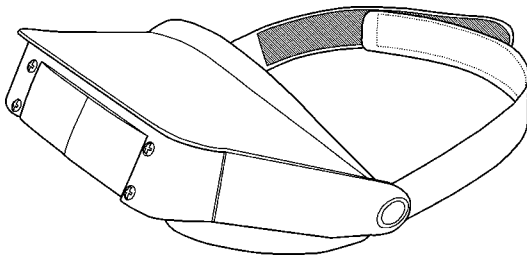

**MANUALE D'USO E MANUTENZIONE**  
**USE AND MAINTENANCE BOOK**  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
**BETRIEBS UND WARTUNGS ANWEISUNGEN**  
**MANUAL DE USO Y MANTENIMIENTO**  
**MANUAL DE USO E MANUTENÇÃO**  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**  
**دليل الإستعمال والرعاية**

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## FEATURES

Head Loupe is a special kind of device for precision magnification. Because of its features, it can be used also by people who normally wear lenses, placing it on them.

Head Loupe allows two kinds of magnification: 2,2x if you use the first system of loupes and 3,3x if you use the second system. The nominal value of the magnification is indicative and not absolute. It is also provided with a head belt in adjustable Velcro and with comfortable anti-reflection coating.

The product or its components cannot be used for purposes different from the one specified in the present manual.

## PRESCRIPTIONS

The product shall be used by qualified medical personnel only, in out-patients' department. It is not conceived for the employment in operation room.

*Do not use to watch directly the sun or other luminous sources!*

*Keep the focal distance of the loupes, in order not to provoke the blurring of the image, which can cause eye weariness.*

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

## FUNCTIONING

At first, it is necessary to block the head loupe with the Velcro stripes of the head belt and then to adjust the inclination of the optical group. If necessary, it is possible to turn it 90° above the forehead (see picture).

*Be sure that the pins supporting the magnification lenses have not been loosened.*

For a correct use, find out the focal distance of the object that you want to magnify, slowly getting nearer to it, until you see it as sharply as possible.

According to your needs, it is possible to choose (see picture) between 2,2x magnification (second lens up) or 3,3x magnification (second lens down); the focal distance varies according to the chosen solution.

*Always use the product at focal distance and not for long time.*

### MAINTENANCE

For a correct use of the product, it is advisable to clean it with soft, anti-scratch fabric and normal glass cleaners, as long as they are not abrasive. Never use solvents.

To disinfect it, it is possible to use disinfection liquids. The product cannot be sterilized in autoclaves. The product must be stored in organic-steam-free environment and at a temperature between +5°C and +30°C.

### GUARANTEE

Thank you for having acquired our product. This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc. No compensation will be given for loss of use of the product.

Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

*This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.*

The guarantee will be invalidated if the registration number has been removed, cancelled or altered.

Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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