



**GIMA**

PROFESSIONAL MEDICAL PRODUCTS

---

***LAMPADA DI WOOD - LAMPADA AD ULTRAVIOLETTI  
PER LA DIAGNOSI E L'ANALISI DELL'EPIDERMIDE  
WOOD'S LAMP - ULTRAVIOLET LIGHT LAMP FOR SKIN DIAGNOSE  
AND ANALYSIS***

### Warning

1. Power supply must be cut off when changing electrical components or maintaining and cleaning machines.
2. The plug must be plugged into an outlet that is properly installed and grounded, to prevent the risk of fire and electric shock.
3. Do not allow children to play with and place the machine in a place where children can avoid touching.
4. Please do not place the machine in damp places such as bathroom to avoid electric shock or other failure.
5. Please do not place the machine in places where painting/coating or oxygen providing during operation.
6. Unless being monitored or instructed by personnel who can be responsible for their personal safety, the people who have physical obstacles, tactile disorders, mental disorders, lacking of experience, common sense or children, are not allow to use the machine.
7. Please contact distributor, service center or related personal with appropriate skill and experience for repairing if the plug is damaged, in case of any dangerous.
8. Maintenance should be processed by trained professionals.
9. This product can only be used by professional beauty practitioners, please pay close attention to working condition of the machine during operating.

### Instruction

Ultraviolet ray lamp is made from general ultraviolet ray which go through the glass-filter with nickel. Wood's lamp was invented by American physical scientist Robert Williams Wood's. The main medical usage is to diagnose the unfavorable condition of skin, such as rash.

### Usage

Beautician use wood's lamp to examine the condition of skin. Different material irradiated with dark purple ray of wood's lamp will appear ray with different colors. The wood's lamp is supposed to be used in a dark room. The purple ray can be used to examine the scarf skin of human body by beautician. What's more, the beautician can do some appropriate skin care for the customer. The invisible skin blemish can be displayed by wood lamp. The displayed shape and color of different skin would be different. For example, if the skin is too thick, the displayed ray should be bright white. The following cases are some of the situation under wood's lamp.

#### Different type of skin under wood's lamp appears different color.

Skin condition	Displayed color
Thick epidermis	White fluorescence
Necrosis cells	White spot
Healthy skin	Blue and white
Water deficiency (thin skin)	Purple
Water deficiency	Light purple
Water abundance	Bright fluorescence
Dark fleck	Brown
Oiled part and pimple	Yellow or pink

The healthy skin on head and hair under wood lamp should appear white and grey purple. The nail should appear bright white. If you doubt that the formula of cosmetic changes, you can put the previous and current cosmetic under the wood lamp for checking. The same formula of cosmetic should appear the same color.

Mostly the wood's lamp are applied on face of human. First, we hook up the sunshade cover on wood's lamp, and then hook up the other end of sunshade cover on customer's face. The operation should be done in dark room.

### Attention

1. While doing the examine, the customer must close their eyes or put cotton cloth on their eyes. Also the beautician are not allowed to stare at the lamp house.
2. The working time couldn't be more than 30 minutes; the tube mustn't be in touch of the skin directly, and shouldn't explore on the customer's skin for long time.



3. Should the powerline was damaged, to avoid danger, you should specify it to the manufacturer or the nominated maintenance department, taking apart and replacing privately is prohibited.

### Technical Parameters

Packing size (cm) : 30x21x7 1pcs  
Packing size (cm): 66x31.5x17.5 6pcs  
Net weight: 1pcs (0.9 Kg); 6pcs (5.4 Kg)  
Gross weight: 1pcs (1.2 Kg); 6pcs (8.6 Kg)  
Voltage: ac 100-120V-/AC 220-240V  
Frequency: 50/60Hz.  
Wattage: 22W



**Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

---

**REF 1004 (GIMA 31200)**

Fabbricante / Manufacturer

**SILVERFOX CORPORATION LIMITED**

No. 18, 1st TongLe Road, TangXia Town, Pengjiang District, JiangMen City, GuangDong Province, China

amy.liang@silverfox.cn

Made in China

Importato da / Imported by:

**Gima S.p.A.**

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

[www.gimaitaly.com](http://www.gimaitaly.com)

