



PROFESSIONAL MEDICAL PRODUCTS

TEST PER LA VISIONE STEREOSCOPICA – TEST FOR STEREOSCOPIC VISION – EXAMEN DE LA VISION STÉRÉOSCOPIQUE – TEST FÜR STEREOSKOPISCHES SEHEN – TEST PARA LA VISIÓN ESTEREOSCÓPICA – TESTE PARA A VISÃO ESTEREOSCÓPICA – ΤΕΣΤ ΣΤΕΡΕΟΣΚΟΠΙΚΗΣ ΟΡΑΣΗΣ –

إختبار للرؤية المجسّمة

Fabbricato da/Manufactured by/Fabriqu  par/
Fabricado por/Hergestellt von/Fabricado por/
Κατασκευασμένο από/مصنوع بواسطة
Levetta s.a.s. di Maffei Michele & C.-Via dei Cancellieri, 6/8
51100 Pistoia - Italy
levetta@levetta.com

Distribuito da/Distributed by/Distrib  par/
Distribuido por/Vertrieben von/Distribuido por/
Διαν μεται από το/موزع من قبل
Gima S.p.A - Via Marconi, 1 - 20060 Gessate (MI) - Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

Made in Italy

REF 5206 (GIMA 31293)



M31293-M-Rev. 3.09.24



ENGLISH

This product is not a Medical Device but it is a purely indicative tool used to help make an initial assessment of the cooperation of the two eyes in forming a single image from the two images produced separately to the brain by each of the two eyes.

It can be used on children from the age of one and on adults.

METHOD AND HOW TO USE:

The test consists of a card featuring 4 pictures: a ball, a fish, a star and a tree. Common pictures were chosen that can be recognised and named both by adults and children from an early age. The patients are asked to indicate what they see on the card and their eye movements are observed. The ball is the only item that is visible with only one eye, whereas the others (each of which is positioned at different depth levels) are only perceived by those with binocular vision and a visual acuity of more than 2-3/10. Cooperating children and adults are shown the test card at a distance of 30 - 40 cm and asked to indicate the pictures: if only the ball is perceived, the test indicates that further ophthalmological investigations are required.

In summary:

Positive result:

all items are identified and named

Doubtful result:

Only one of the pictures is identified

Negative result:

None of the three-dimensional items are identified.

INDICATIONS FOR TEST EVALUATION:

Tool used for the initial assessment of suspected strabismus, amblyopia, visual reduction in one or both eyes, or as a screening test for stereoscopic vision. Uncooperative children can be shown the test to observe whether they try to grab one of the items besides the ball (fish, star, tree). The test is normal is this happens. If this does not happen, the test cannot be considered certainly pathological and can be repeated at a later date.

Important: in the event of a negative or doubtful result, a complete eye examination is recommended. The results of the test are indications; it is necessary to refer to a specialist doctor for further diagnosis and verification.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.