



Jiangsu Suyun Medical Materials Co., Ltd 江苏苏云医疗器材有限公司

Add: No.1, Medicine Lane, Renmin Road, Lianyungang, Jiangsu Province, China
地址: 中国江苏省连云港市人民路制药巷1号 (222002)

STERILE OTOLARYNGOLOGICAL SET FOR SINGLE USE

Components

1pcs plastic nasal speculum, 2pcs (adult and pediatric size) plastic ear funnels, 1pcs plastic tongue depressor, 1pcs plastic forceps, 6pcs cotton tip applicators, 2pcs plastic laryngeal mirrors and 1pcs napkin.

Product usage

It is widely used in the ENT clinic examination, nasal speculum examines the nose, ear funnel could watch the inner ear, tongue depressor and plastic laryngeal mirrors examine the mouth and throat. Cotton tip applicator used for sampling and cleaning. Before operation, firstly check if package have broken, then examine whether all components are enough in the plate, then fix the napkin at the chest of people, to avoid pollution.

Attention

Do not use if the package broken. Do not reuse it.
Operation should be generally, so that the patient will not feel hurt.

Storage

Products should be stored in cool, dry, clean environment.

Expiration date

5 years after sterilization.

Index of Symbols

	Product code		Lot number		Disposable device, do not re-use
	Manufacturer		Expiration date		Do not re-sterilize
	Authorized representative in the European community		Medical Device complies with Directive 93/42/EEC		Don't use if package is damaged
	Sterilized using ethylene oxide		Date of manufacture		

REF 488802 (31457)



Jiangsu Suyun Medical Materials co., Ltd.
No. 1 Medicine Lane, Renmin Road,
222002 Lianyungang,
PEOPLE'S REPUBLIC OF CHINA
Made in China



Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße, 80, 20537, Hamburg, Germany

Importato da / Imported by:

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

江苏苏云医疗器材有限公司
JIANGSU SUYUN MEDICAL MATERIALS CO., LTD.
秦宏平

STERILE EO

0123

