


THE LIGHTED EAR CURETTE™ WITH MAGNIFICATION - USER INSTRUCTIONS
DIE LIGHTED EAR CURETTE™ MIT VERGRÖSSERUNG - GEBRAUCHSANWEISUNG
LIGHTED EAR CURETTE™ CON AUMENTO - INSTRUCCIONES DE USO
LIGHTED EAR CURETTE™ AVEC GROSSISSEMENT - MODE D'EMPLOI
LA LIGHTED EAR CURETTE™ CON INGRANDIMENTO - ISTRUZIONI PER L'USO
A LIGHTED EAR CURETTE™ COM AMPLIAÇÃO - INSTRUÇÕES PARA O UTILIZADOR

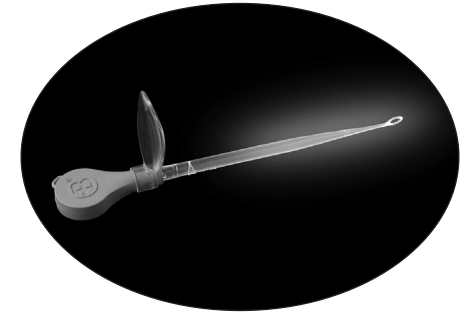

B **BIONIX**®

+1 419 727 8997

www.Bionix.com

©2016 Bionix Development Corp.
Rev. 12/5/2016 • RM95-XXXX • Patented



EC REP

Obelis s.a • Boulevard Général Wahis 53
1030 Brussels, BELGIUM • Tel: +(32) 2.732.59.54
Fax: +(32) 2.732.60.03 • E-Mail: mail@obelis.net



Bionix Development Corp. • 5154 Enterprise Blvd.
Toledo, OH, 43612 • U.S.A. • Phone: 800-551-7096
Fax: 800-455-5678 • www.Bionix.com

THE LIGHTED EAR CURETTE™ WITH MAGNIFICATION - USER INSTRUCTIONS

Assembly/Disassembly:

1. To assemble the lighted ear curette, insert the end of the curette into the light source by aligning the pegs on the curette end with the grooves inside the light source. Push down and twist FIRMLY clockwise until rotation stops. This will lock the curette into place and activate the light. NOTE: This product is a spring-loaded device; improper assembly may cause ejection of the ear curette.
2. To attach the magnification lens, snap the lens onto the neck of the light source so that the top of the lens angles towards the ear curette tip.
3. To disassemble, push down on the curette and rotate counter clockwise to remove the curette from the light source. The light will go off when the curette is removed.

Instructions For Cerumen Removal

The intended use of the Lighted Ear Curette is to remove cerumen from the ear canal.

1. Use an otoscope to examine the ear canal and determine cerumen location.
2. Select the appropriate style lighted ear curette for the type of patient and cerumen occlusion. Assemble according to instructions.
3. Use lighted ear curette to remove cerumen from the ear canal. CAUTION: DO NOT hold device by light source as accidental ejection of curette may occur if not assembled properly.
4. Repeat otoscope exam. Determine location of any remaining cerumen.
5. Repeat curettage procedure until sufficient cerumen is removed to allow adequate visualization of the tympanic membrane.
6. Disassemble the lighted ear curette, and discard the used curette in the usual fashion. Save the light source for the next curettage procedure. LEC Tips are intended for single patient use only.