

# Disposable Nasal Speculum

## INSTRUCTIONS FOR USE



Obelis s.a  
Boulevard Général Wahis 53  
1030 Brussels, BELGIUM  
Tel: +(32) 2.732.59.54  
Fax: +(32) 2.732.60.03  
E-Mail: mail@obelis.net

**Rx ONLY**



Bionix LLC  
1670 Indian Wood Circle  
Maumee, Ohio 43537, United States  
Phone: +1 419.727.8421  
Fax: +1 800.455.5678  
www.Bionix.com

**MADE IN THE U.S.A.**

Importer, Importeur, Importör,  
Importador, Importateur,  
Εισαγωγέας, Importeur,  
Maahantuoja, Importatore:



**MedEnvoy**

Prinses Margrietplantsoen 33 - Suite 123  
2595 AM The Hague  
The Netherlands

This manual applies to the **REF** numbers indicated below:

#9878 - 20 per Box

#9877 - 48 per Box



©2021 Bionix LLC | 1670 Indian Wood Circle  
Maumee, Ohio 43537 | United States  
Phone: 800.551.7096 | Fax: 800.455.5678  
Email: bionix@bionix.com  
www.bionix.com | RM95-0028 | Rev. D

## ENGLISH

### DISPOSABLE NASAL SPECULUM - USER INSTRUCTIONS

**Intended Use:** The intended use of the Disposable Nasal Speculum is to allow direct examination of the anterior part of the nose.

**Indications for Use:** The Disposable Nasal Speculum is indicated for use in diagnosis, extraction of foreign bodies, and nasal packing by healthcare professionals on all patient populations.

#### Instructions:

To use, hold the Disposable Nasal Speculum comfortably in one hand. Position the patient in a manner that ensures proper access and adequate lighting. Gently insert the Disposable Nasal Speculum tip into the patient's nares, with the blades in the "closed" position. Slowly squeeze the handle to spread the Disposable Nasal Speculum blades and open the nares. Stop when adequate exposure has been obtained.

**⚠ CAUTION:** Discontinue use of the Disposable Nasal Speculum if the patient experiences pain or trauma to the nose or the nasal mucosa.

**⚠ WARNING:** Cross-contamination risk. Do not reuse the Disposable Nasal Speculum as this may spread contamination from one patient to another patient.

**Medical Device Reporting:** Notice to Users and/or Patients in EU: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Clinical Benefits:** Indicated in Intended Use.

**Contraindications:** There are no absolute contraindications related to the use of the Disposable Nasal Speculum.

**Residual Risk:** Risk associated with the use of this product has been reduced as far as possible, but the product cannot completely eliminate potential patient or user harm arising from the following:

- Harm from mechanical hazards
- Harm from misuse, or use error
- Harm from unanticipated origins

	Medical Device		Do Not Use if Package is Damaged
	Manufacturer		Do Not Reuse Single-Use Device
	Date of manufacture		Single patient-multiple use
	Use-By Date		Consult Instructions For Use (IFU)
	Lot Code		Caution
	Reorder Number		Warning
	Serial number		Authorized representative in the European Community
	Sterilized using irradiation		European Conformity
	Do Not Resterilize		Prescription Only or "For Use by or on the order of a licensed medical professional"
	Non-sterile		Temperature limitation