HEINE BETA 200®

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Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries).

We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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Before using the Ophthalmoscope HEINE BETA 200® read these instructions carefully and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information

Range of applications: This instrument is designed for examination of the eye.

Safety in use: Use only handles or devices which are specifically designed for medical use as a power supply. The supply voltage must be the same as bulb voltage.

Please note: The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries or alkali manganese batteries are used.

Key to Illustrations

- Spectacle rest
- Dustproof viewing aperture
- Lens wheel
+dpt: 1-10 | 15 | 20 | 40
-dpt: 1-10 | 15 | 20 | 25 | 35
- Diopter readout (plus = green, minus = red)
- Thumbrest
- HEINE XHL® Xenon halogen bulb
- Locating pin
- Dustproof window
- Aperture wheel
- Filter selector (an interference red-free filter can be combined with any aperture)

Using the instrument

To ensure maximum user comfort, hold the instrument so that the thumb is located on the thumbrest (5). This makes it possible to operate the lens wheel (3), aperture wheel (9), and filter selector (10).

Changing the bulb

Please note: The performance of this instruments can only be guaranteed if genuine HEINE bulbs are used.

Simply detach the ophthalmoscope from the handle and pull the bulb out. Clean the glass of the new bulb with a soft cloth.

Insert the bulb fully into the body of the instrument. The pin must locate in the groove.

Cleaning

The instrument housing can be cleaned with a cloth moistened with alcohol. Clean the glass of (2) and (8) and the diopter readout (4) with a cotton-wool bud moistened with alcohol. The tip should be rotated without excessive pressure in a spiral from the centre to the edge of the window.

Gekauft bei / Bought from / Acheté chez / Comprado a / Acquistato presso

am / date / le / el / il