
**HEINE** QUALITY  
MADE IN GERMANY

# HEINE K 180<sup>®</sup> Ophthalmoscope

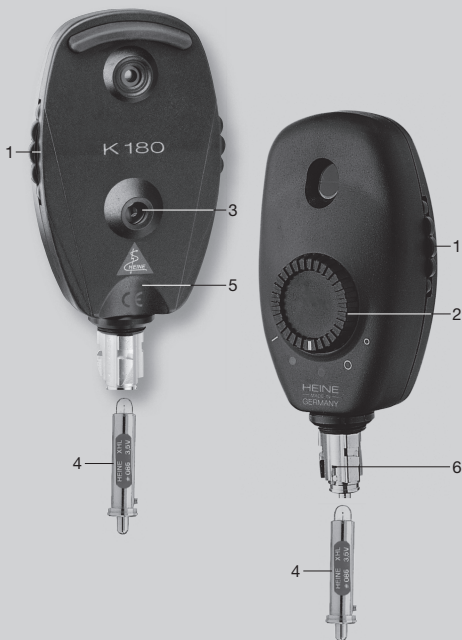CE 93/42/EWG/CEE/EEC

**HEINE Optotechnik GmbH & Co. KG**  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49(0)81 52/38-0  
Fax +49(0)81 52/38-202  
E-Mail: [info@heine.com](mailto:info@heine.com) · [www.heine.com](http://www.heine.com)  
med 7896 1/9.11

# Instructions

## HEINE K180® Ophthalmoscope

Before using the ophthalmoscope read these instructions carefully and keep them in a safe place for future reference.

 In accordance with local regulations this product should be disposed of as an electronic device separately .

### Safety information

**Range of applications:** The K 180 ophthalmoscope is designed solely for examination of the eye.

**Safety in use:** Use only handles or devices which are specifically designed for medical use as a power supply. The supply voltage must be the same as bulb voltage.

### Using the instrument

For maximum comfort during prolonged use, hold the instrument so that the thumb coincides with the groove in the instrument body (5). In this position, the lens (1) and filter wheel (2) can be operated comfortably with the forefinger of the same hand. The value of the chosen correction can be seen in the window (3). Positive values are green, negative values red.

### Cleaning and Sterilisation

The ophthalmoscope body can be wiped clean with a soft cloth soaked in alcohol. The glass discs in the viewing and light-exit windows can be cleaned by using a cotton wool swab in a spiral motion from the centre of the disc working towards the edge.

### Changing the bulb

Disconnect the ophthalmoscope from the handle and pull out the bulb (4). Wipe clean the glass of the new bulb and push the bulb all the way in. The projection (7) must locate in the slit (6).

**Please note:** The performance of this unit can only be guaranteed if genuine HEINE XHL Xenon Halogen spare bulbs are used.

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## Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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